



(albuterol sulfate 117 mcg)
Inhalation Powder



(fluticasone propionate
113 mcg and salmeterol
14 mcg) Inhalation Powder



(fluticasone propionate
113 mcg) Inhalation Powder

Welcome to the Digihaler[®] Dashboard

Inhaler event data in one accessible place

The Digihaler Dashboard is a secure, online tool that enables you to view patients' inhaler event data. **If the patient consents to share**, data is sent from the Digihaler Mobile App to the Dashboard, which may allow healthcare providers to have more informed treatment discussions with their patients.^{1,2}

The Digihaler Dashboard helps you to:

- Securely access data from a single patient or multiple patients
- See when patients use their ProAir Digihaler, AirDuo Digihaler, and/or ArmonAir Digihaler, and assess inspiratory flow rates
- Access daily, weekly, or monthly reports

With inhaler event information accessible to you and your team,
you may be able to make more informed treatment decisions for your patients.^{1,2}

INDICATIONS FOR PROAIR[®] DIGIHALER[®]

ProAir[®] Digihaler[®] (albuterol sulfate) Inhalation Powder is indicated in patients ≥ 4 years of age for the treatment or prevention of bronchospasm with reversible obstructive airway disease and in patients ≥ 4 years of age for the prevention of exercise-induced bronchospasm.

IMPORTANT SAFETY INFORMATION FOR PROAIR[®] DIGIHALER[®]

- **Contraindications:** ProAir Digihaler (albuterol sulfate) Inhalation Powder is contraindicated in patients with hypersensitivity to albuterol or patients with a severe hypersensitivity to milk proteins. Rare cases of hypersensitivity reactions, including urticaria, angioedema, and rash have been reported after the use of albuterol sulfate. There have been reports of anaphylactic reactions in patients using inhalation therapies containing lactose

INDICATIONS FOR ARMONAIR[®] DIGIHALER[®] AND AIRDUO[®] DIGIHALER[®]

ArmonAir[®] Digihaler[®] (fluticasone propionate) inhalation powder is indicated for the maintenance treatment of asthma as prophylactic therapy in patients 12 years of age and older.

Limitation of Use: ArmonAir Digihaler is not indicated for the relief of acute bronchospasm.

AirDuo[®] Digihaler[®] (fluticasone propionate and salmeterol) inhalation powder is indicated for the treatment of asthma in patients aged 12 years and older. AirDuo Digihaler is only for patients uncontrolled on an inhaled corticosteroid (ICS) or whose disease severity clearly warrants an ICS/Long-acting beta2-agonist (LABA).

Limitation of Use: AirDuo Digihaler is not indicated for the relief of acute bronchospasm.

IMPORTANT SAFETY INFORMATION FOR ARMONAIR[®] DIGIHALER[®] AND AIRDUO[®] DIGIHALER[®]

- **Contraindications:** ArmonAir Digihaler and AirDuo Digihaler are contraindicated in:
 - » Primary treatment of status asthmaticus or other acute episodes of asthma requiring intensive measures
 - » Patients with known severe hypersensitivity to milk proteins or any ingredients of ArmonAir Digihaler or AirDuo Digihaler

Please see additional Safety Information throughout this brochure and full Prescribing Information for [ProAir Digihaler](#), [ArmonAir Digihaler](#) and [AirDuo Digihaler](#).

Table of Contents

For All Dashboard Users

Section 1: Account Basics

- The Dashboard Team
- Signing Up
- Joining Multiple Programs
- Logging Out
- Resetting Your Password

Section 2: Inviting Patients

- Adding Patients to the Dashboard
- Inviting Patients

Section 3: Navigating the Patient List

- Viewing the Patient List
- Filtering the Patient List
- Removing Patients from your Program

Section 4: Viewing Patient Details

- Patient Details
- Understanding the Patient Details Screen
- The Digihaler® Events Chart
- The Recorded Digihaler® Events Table
- Digihaler® Connectivity
- Accessing Detailed Data
- The Peak Inspiratory Flow Chart
- The Inhalation Volume Chart

Section 5: Exporting Patient Data

- Setting up an Export
- Applying Filters

Section 6: Support and Troubleshooting

For Dashboard Admins and Super Admins

Section 7: Inviting Teammates

- Managing your program together
- Changing the Role of a Teammate
- Managing Pending Invitations
- Exporting an Audit Log

Section 8: Program Settings

- Program Settings
- Configuring Filters
- Using Filters

Section 1: Account Basics

The Dashboard Team (1 of 2)

You will have a program specific to your department within the Digihaler® Dashboard. Your program will be run by a team made up of three different types of users who are responsible for specific administrative roles.

Super Administrator

Primary responsibilities:

- Requesting Dashboard access from a Teva Representative
- Inviting new Super Administrators, Administrators, and Dashboard Users—Super Administrators should choose roles that are the best fit for each member
- Inviting patients to participate in the program
- Removing team members if needed
- Changing existing users' roles if needed

***Please Note** that there can be multiple Super Administrators, since each healthcare facility is different in structure and process. For instance, a Super Administrator can be a member of the facility's IT department, as well as a head doctor of a small practice.*

Administrator

Primary responsibilities:

- Assisting the Super Administrators in managing the team's specific program
- Inviting, removing, and managing other Administrators and Dashboard Users
- Identifying, inviting, and adding patients to Patient List
- Promoting Dashboard Users to Admins if needed

Dashboard User

Primary responsibilities:

- Patient care—note that Dashboard Users have full access to inhaler event data and will likely be a healthcare provider
- Inviting and removing patients
- Viewing Patient List
- Reviewing and assessing patient inhaler event data

The Dashboard Team (2 of 2)

Here is a comparison chart of the roles and permissions for each team member:

| | Super Administrator | Administrator | Dashboard User |
|-------------------------|---------------------|---------------|----------------|
| View Patient List | ● | ● | ● |
| View Patient Details | ● | ● | ● |
| Invite Patient | ● | ● | ● |
| Remove Patient | ● | ● | ● |
| View Team List | ● | ● | |
| Invite Super Admin user | ● | | |
| Invite Admin user | ● | ● | |
| Invite Dashboard user | ● | ● | |
| Remove Admin user | ● | | |
| Remove Dashboard user | ● | ● | |
| Using Filters | ● | ● | ● |
| Configuring Filters | ● | ● | |
| Download Audit Log | ● | ● | |

Signing Up (1 of 6)

- 1 You will be sent an invitation link by one of the members on your team.


Super Administrator

Super Admins are invited by a Teva representative or other Super Admins.

Administrator/Dashboard User

Admins and Dashboard Users are invited by an Admin or a Super Admin.

- 2 Your invitation link will direct you to the Digihaler® Dashboard's landing page (<https://us.dhp.ehealth.teva/careprogram/welcome>)

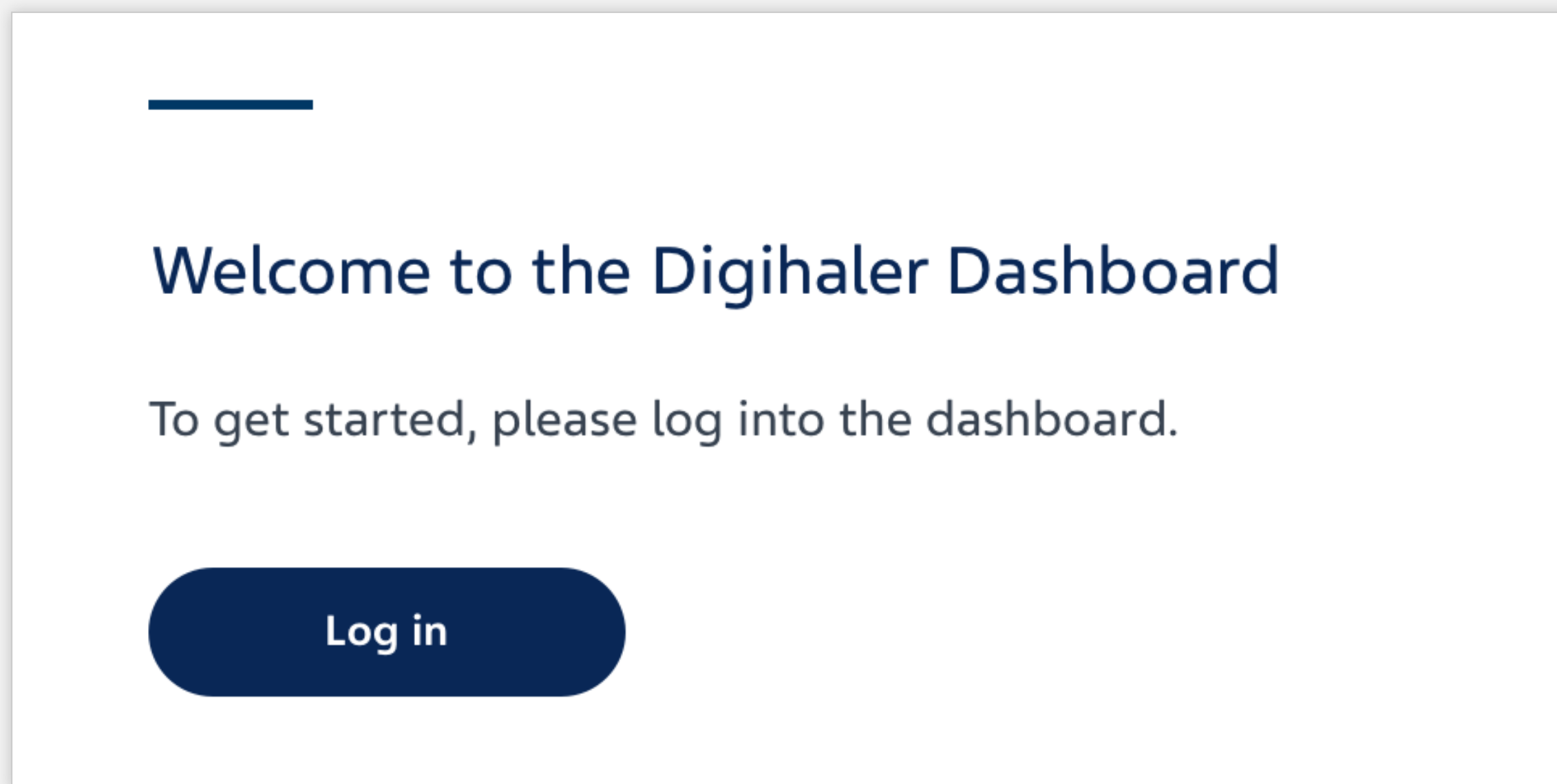

Signing Up (2 of 6)

- 3 Click **“Login”** where you’ll be directed to a login screen
 - a. For your first time sign-in, click “Sign Up.”
 - b. If it is not your first time, type in the email you have associated with your account and the password you have already created.


Login




Log In

New to Digihaler Dashboard? [Sign Up](#)

Email

Password 

Remember me 


Only select this option on your private or personal device. Do not use this if you are using a public or shared computer.

Login

[Forgot Password](#)

Signing Up (3 of 6)

4 Click on "Sign Up" to create your login credentials

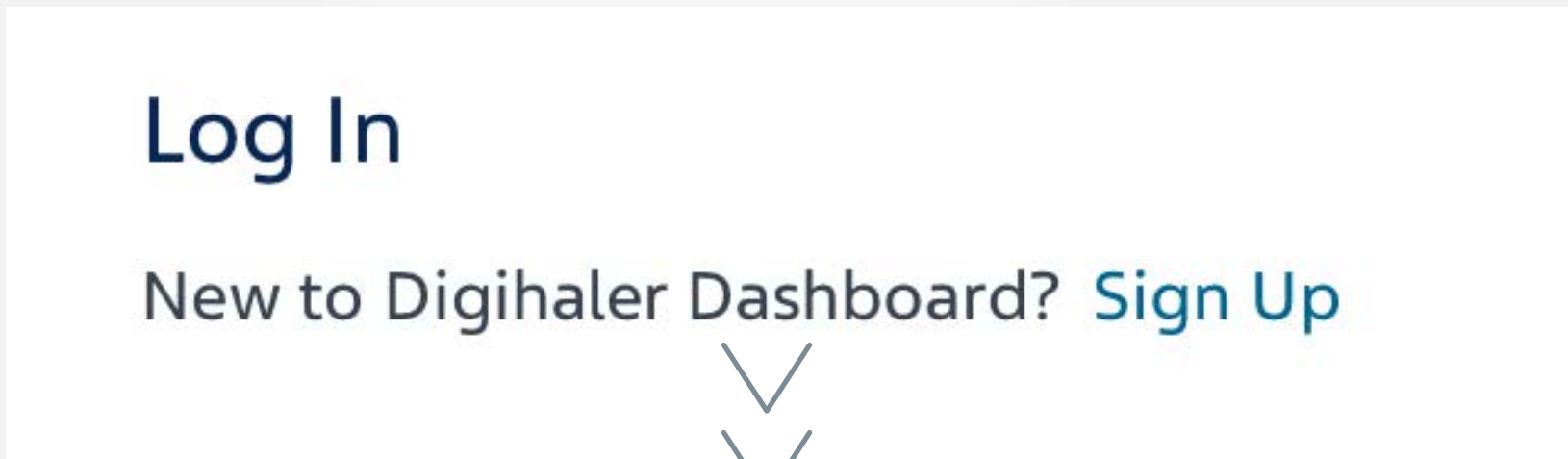A screenshot of a "Sign Up" form. The form is titled "Sign Up" and includes a link for "Already have an account? Log In". The form contains the following fields:

- *First Name: Karen
- *Last Name: Demo
- *Email: example@email.com (with a note: "This will also be your username.")
- *Confirm Email: example@email.com
- *Password: A field with 8 dots representing a password, with a note: "Your password must be alphanumeric, mix upper and lowercase letters, and must contain at least 8 characters." and a toggle icon for visibility.

A dark blue "Sign up" button is located at the bottom of the form.

Signing Up (4 of 6)

5 Your invitation link will direct you to the Digihaler® Dashboard's landing page. You'll be prompted to:

- a. Sign up using your first name, last name, email, and confirm email

Please Note that you must use the same email that your inviter has specified

- b. Create a password using a minimum of 8 characters, consisting of both uppercase and lowercase letters, and at least one number
- c. Click "Sign up"


The screenshot shows a 'Sign Up' form with the following fields and callouts:

- Callout a:** Points to the 'First Name' field containing 'Karen'.
- Field:** 'Last Name' containing 'Demo'.
- Field:** 'Email' containing 'example@email.com'. Below the field is the text: 'This will also be your username.'
- Field:** 'Confirm Email' containing 'example@email.com'.
- Callout b:** Points to the 'Password' field, which is currently masked with dots. Below the field is the text: 'Your password must be alphanumeric, mix upper and lowercase letters, and must contain at least 8 characters.'
- Callout c:** Points to the 'Sign up' button.


Signing Up (5 of 6)

- 6 The next screen will prompt you to enter a One-Time Sign-in code. The code will be emailed to you to confirm that the email you have entered is valid.

A 6-digit code will be sent to the email you have associated with the program. Check your email for a message from “Teva Pharmaceuticals.” If you do not see the email in your inbox, check your spam folder.

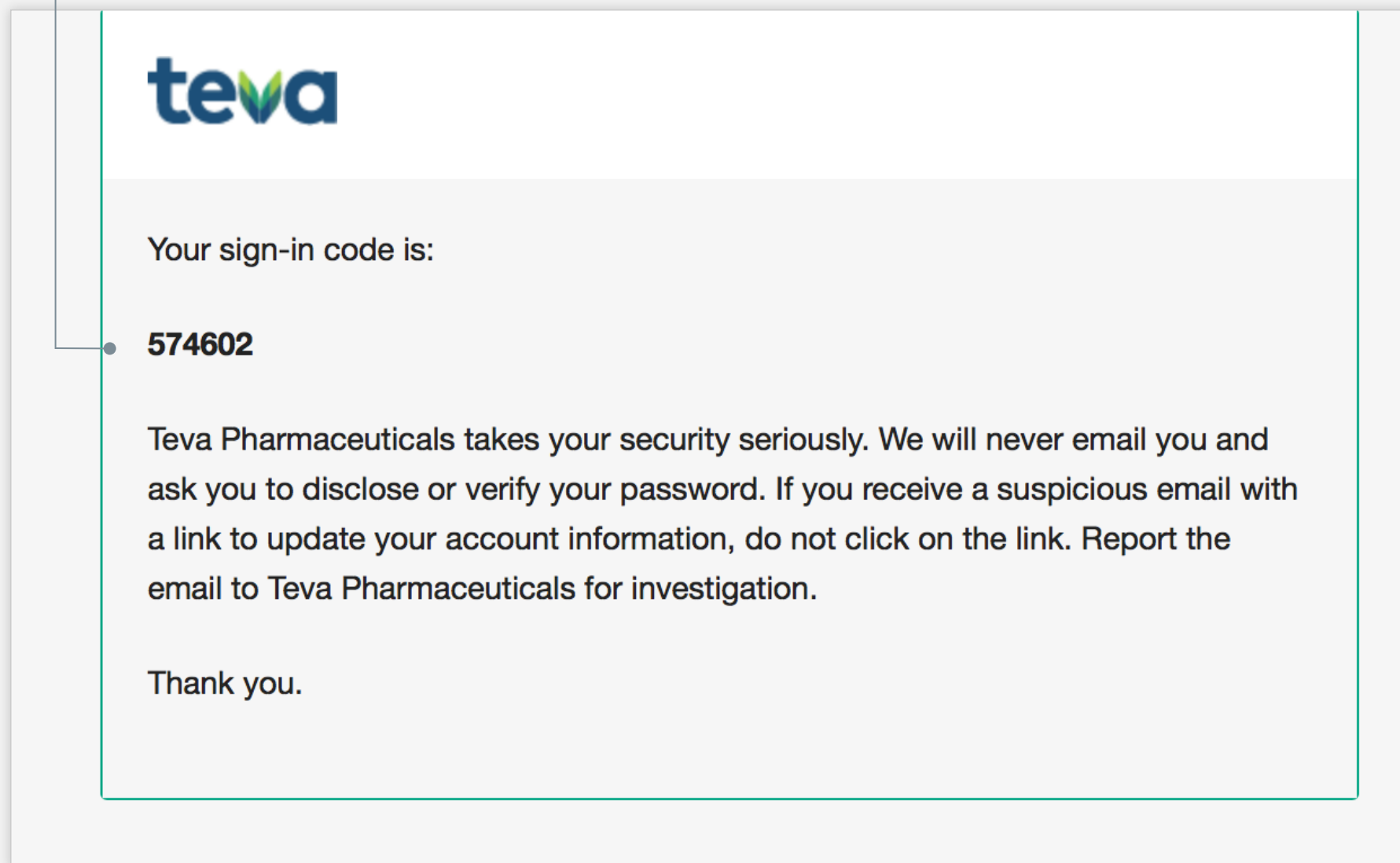

The email will contain a code for you to enter into the program screen's form field as shown here

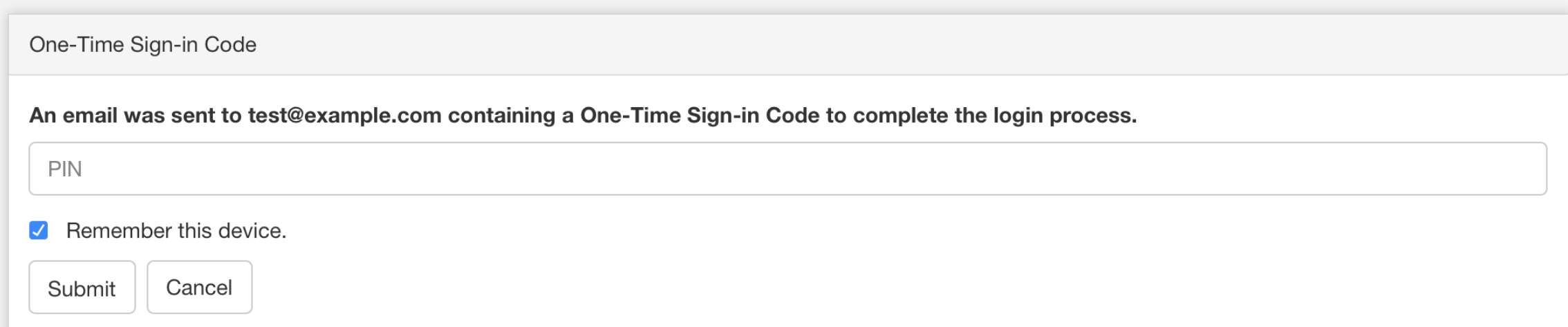
A screenshot of a login form. At the top, it says "One-Time Sign-in Code". Below that, a message reads: "An email was sent to test@example.com containing a One-Time Sign-in Code to complete the login process." There is a text input field labeled "PIN" with the placeholder text "PIN". Below the input field is a checked checkbox labeled "Remember this device." At the bottom of the form are two buttons: "Submit" and "Cancel".

Please Note

- You must enter the 6-digit code with no additional spaces. Entering a mistaken code will result in an expired PIN error code, and you will be sent a new one.
- The PIN expires after 10 minutes. You will no longer need the PIN for future logins if using the same device and browser.

Signing Up (6 of 6)

- 7 Once you complete the sign-up process, you will be prompted to join the program you were added to.

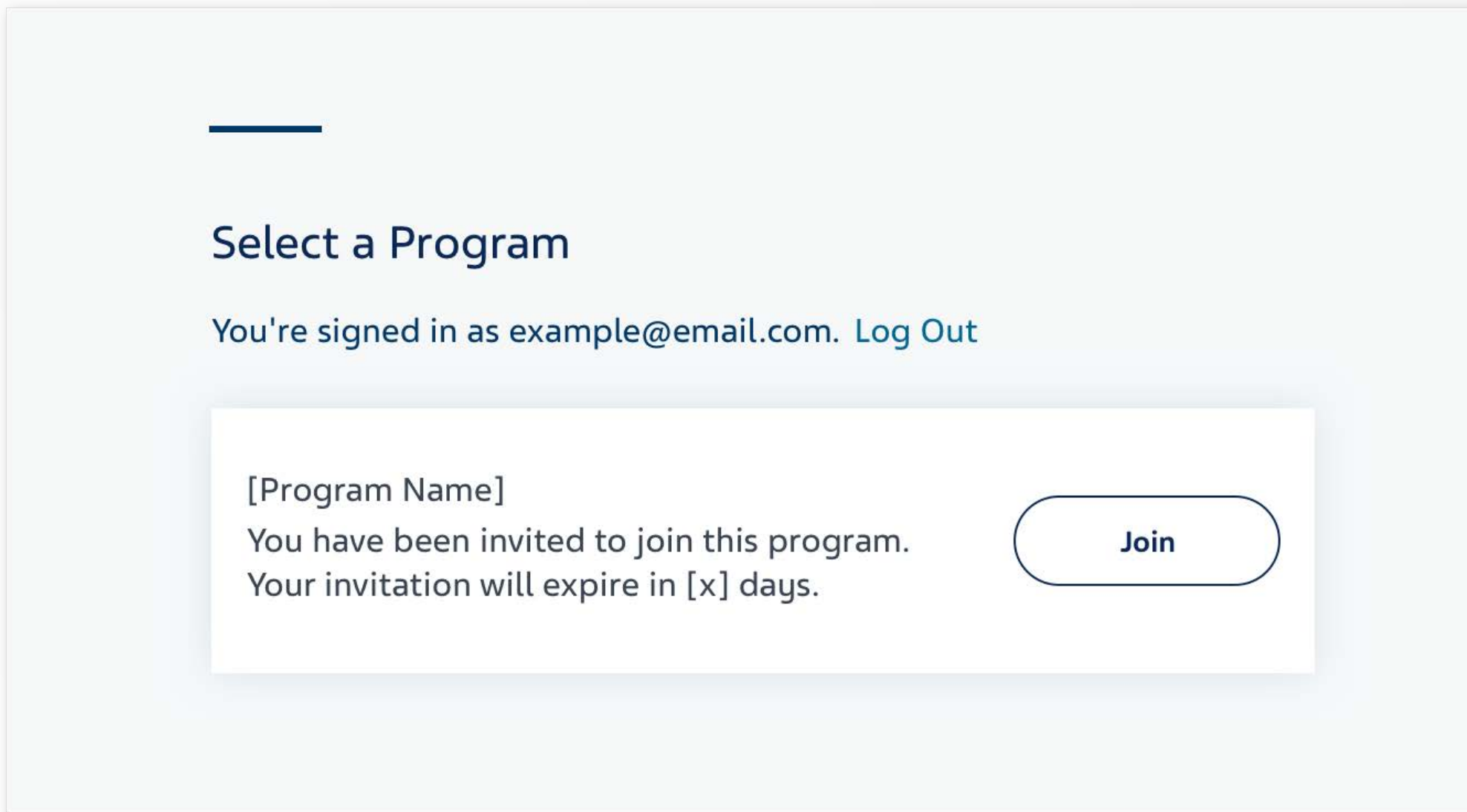

For Super Admins

Once the Super Administrator has created the program and set up their login credentials, they can begin to invite Administrators and Dashboard Users

For Admins

Once Administrators have set up their login credentials, they can begin to invite Dashboard Users or patients

For Dashboard Users

Once Dashboard Users have set up their login credentials, they can begin to invite patients and view their data after the patient consents to share via the app.

Joining Multiple Programs (1 of 2)

You may be invited to multiple programs if you are a part of more than one practice, or charged with managing more than one team.

Your sign up process will follow the same instructions as above, but you will be prompted to choose the program you would like to view immediately after sign-in.

Select a Program

You're signed in as example@email.com. [Log Out](#)


| | |
|---|--------|
| [Program Name] | Select |
| [Program Name] | Select |
| [Program Name] You have been invited to join this program. Your invitation will expire in [x] days. | Join |

If you are invited to another program, you will see upon login, the additional program with a "Join" button.

Joining Multiple Programs (2 of 2)

If you want to switch programs or join another program **while already logged into the Dashboard**, click the down arrow next to the program name in the left hand navigation menu and select the program you'd like to switch to, or select "Pending Invitations" to join a program to which you have been invited.

Click the down arrow next to the program name in the left hand menu.

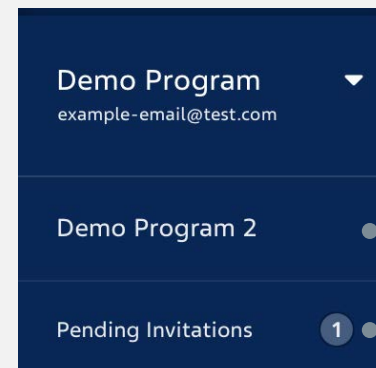

Select program or "Pending Invitations" to join a program.

Patient List Invite Patient

10 Patients Filter Export Data 1 - 10 of 10

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|------------------|------------|--------------------------------|--------|-------------------|---------------------------------|--------|----------------|-----------------------------------|--------|-------------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1958 | 40 | 38% | 30 minutes ago | 22 | 61% | 1 hour ago | - | - | - |
| Demo, Patient 2 | 04/26/1964 | 17 | 100% | 5 hours ago | - | - | - | 44 | 100% | 5 hours ago |
| Demo, Patient 3 | 03/22/1966 | 12 | 83% | 23 hours ago | 33 | 97% | 3 hours ago | - | - | - |
| Demo, Patient 4 | 11/14/1978 | 5 | 60% | 20 minutes ago | 43 | 93% | 20 minutes ago | - | - | - |
| Demo, Patient 5 | 02/06/1947 | 4 | 0% | 30 minutes ago | - | - | - | 37 | 97% | 15 minutes ago |
| Demo, Patient 6 | 01/11/1984 | 3 | 0% | a few seconds ago | - | - | - | 78 | 64% | a few seconds ago |
| Demo, Patient 7 | 09/05/1957 | 0 | - | 3 hours ago | 10 | 30% | 3 hours ago | 10 | 30% | 3 hours ago |
| Demo, Patient 8 | 10/24/1999 | 0 | - | 3 hours ago | 10 | 30% | 3 days ago | 0 | - | 3 days ago |
| Demo, Patient 9 | 04/16/1950 | 0 | - | 4 days ago | 10 | 30% | 4 days ago | 0 | - | 1 month ago |
| Demo, Patient 10 | 02/04/1948 | 0 | - | 3 days ago | 0 | - | 2 days ago | 0 | - | 2 months ago |

Version 4.3.0 © 2022 Teva Respiratory, LLC teva

Logging Out

To log out, simply select “Logout” from the bottom of the left hand navigation menu. You will be redirected to the landing login page.

You will be automatically logged out after 30 minutes of inactivity.

The screenshot shows the 'Patient List' page in a web application. The left sidebar contains a navigation menu with 'Log Out' at the bottom. The main content area displays a table of 10 patients with columns for 'Last, First', 'DOB', and three medication categories: 'ProAir Digihaler (Last 7 Days)', 'AirDuo Digihaler (Last 30 Days)', and 'ArmonAir Digihaler (Last 30 Days)'. Each medication category has sub-columns for 'Events', '% Good', and 'All Connected'. The table data is as follows:

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|------------------|------------|--------------------------------|--------|-------------------|---------------------------------|--------|----------------|-----------------------------------|--------|-------------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1958 | 40 | 38% | 30 minutes ago | 22 | 61% | 1 hour ago | - | - | - |
| Demo, Patient 2 | 04/26/1964 | 17 | 100% | 5 hours ago | - | - | - | 44 | 100% | 5 hours ago |
| Demo, Patient 3 | 03/22/1966 | 12 | 83% | 23 hours ago | 33 | 97% | 3 hours ago | - | - | - |
| Demo, Patient 4 | 11/14/1978 | 5 | 60% | 20 minutes ago | 43 | 93% | 20 minutes ago | - | - | - |
| Demo, Patient 5 | 02/06/1947 | 4 | 0% | 30 minutes ago | - | - | - | 37 | 97% | 15 minutes ago |
| Demo, Patient 6 | 01/11/1984 | 3 | 0% | a few seconds ago | - | - | - | 78 | 64% | a few seconds ago |
| Demo, Patient 7 | 09/05/1957 | 0 | - | 3 hours ago | 10 | 30% | 3 hours ago | 10 | 30% | 3 hours ago |
| Demo, Patient 8 | 10/24/1999 | 0 | - | 3 hours ago | 10 | 30% | 3 days ago | 0 | - | 3 days ago |
| Demo, Patient 9 | 04/16/1950 | 0 | - | 4 days ago | 10 | 30% | 4 days ago | 0 | - | 1 month ago |
| Demo, Patient 10 | 02/04/1948 | 0 | - | 3 days ago | 0 | - | 2 days ago | 0 | - | 2 months ago |


Select "Logout" from left hand navigation menu

- About
- Help
- Privacy Notice
- Terms of Use
- Audit Log
- Log Out


Resetting Your Password

If you've forgotten your password:

- a. Select "Forgot Password" from the main Login screen.
- b. Enter the email associated with your Dashboard account to start the password reset process.

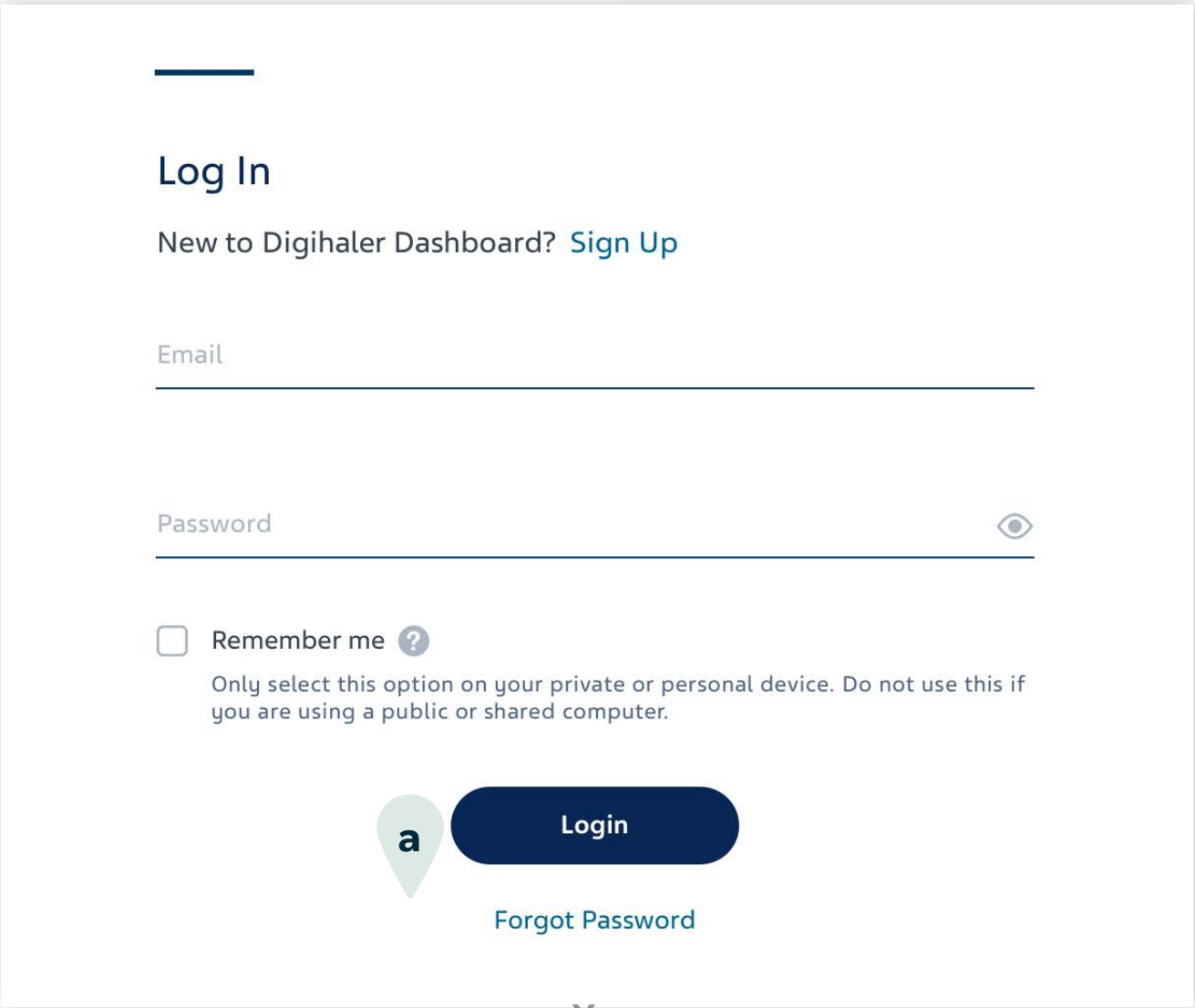

The screenshot shows the 'Log In' page. At the top, it says 'Log In' and 'New to Digihaler Dashboard? Sign Up'. Below are input fields for 'Email' and 'Password'. There is a 'Remember me' checkbox with a help icon and a note: 'Only select this option on your private or personal device. Do not use this if you are using a public or shared computer.' At the bottom, there is a dark blue 'Login' button and a 'Forgot Password' link. A callout bubble with the letter 'a' points to the 'Forgot Password' link.

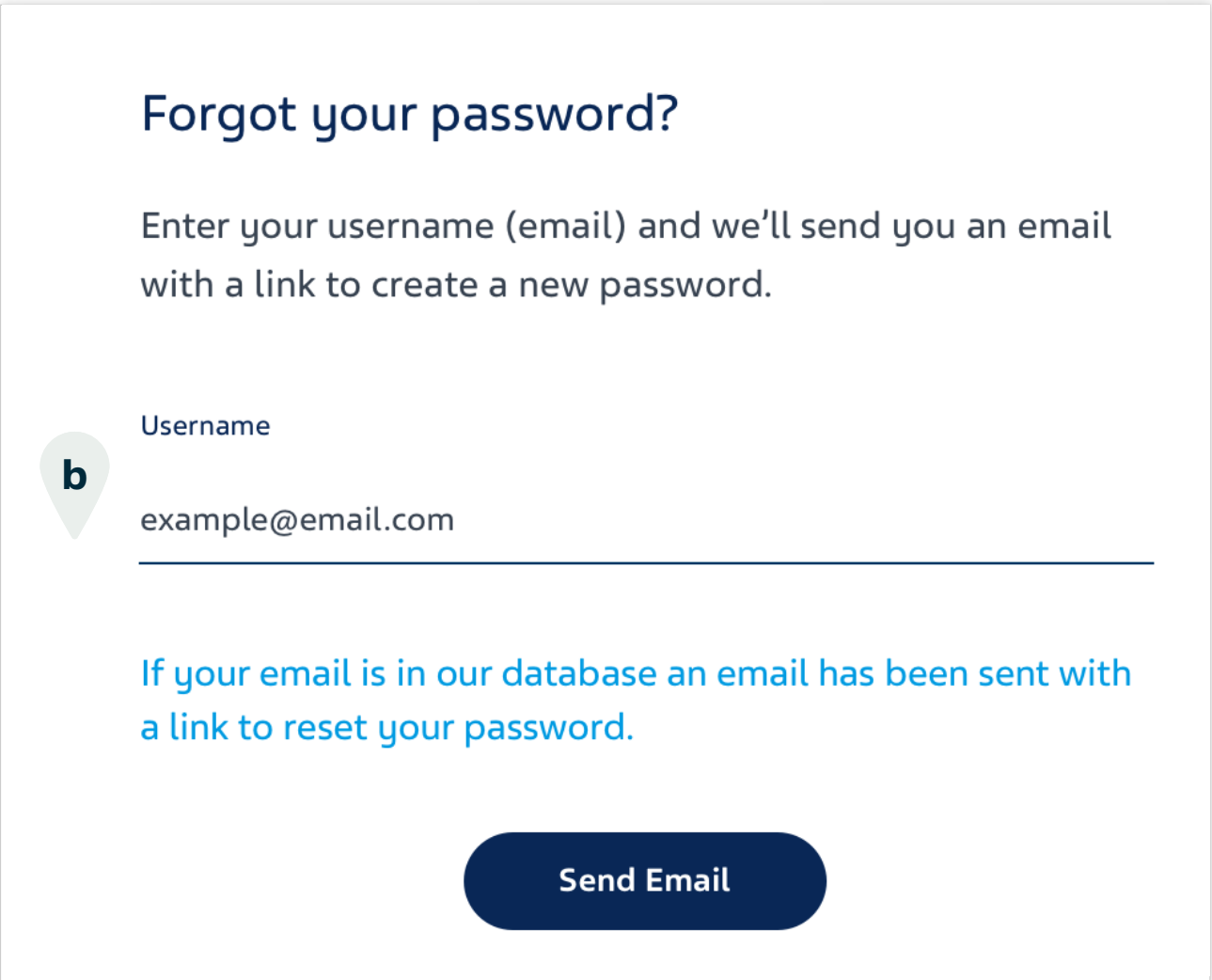

The screenshot shows the 'Forgot your password?' page. It asks the user to 'Enter your username (email) and we'll send you an email with a link to create a new password.' Below is a 'Username' input field with the example 'example@email.com'. A callout bubble with the letter 'b' points to the input field. At the bottom is a dark blue 'Send Email' button.

An email will be sent to your email account. Click the link provided in the email and you will be directed to a new login screen where you can enter a new password and confirm. Your password will be reset for all future logins.

Section 2: Inviting Patients

Adding Patients to the Dashboard

The Digihaler® Dashboard allows doctors and patients to work together on care. When patients choose to share their inhaler event data with their healthcare provider, they can review this information together during consultations. **The information provided by the Digihaler Dashboard may support more informed conversations about treatment.**^{1,2}

The Digihaler Dashboard incorporates industry-standard security and privacy protections.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Paradoxical Bronchospasm:** ProAir Digihaler can produce paradoxical bronchospasm that may be life-threatening. Discontinue ProAir Digihaler and institute alternative therapy if paradoxical bronchospasm occurs

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Serious Asthma-Related Events – Hospitalizations, Intubations, Death:** Use of a LABA as monotherapy (without an ICS) for asthma is associated with an increased risk of asthma-related death. Available data from controlled clinical trials also suggest that use of LABA as monotherapy increases the risk of asthma-related hospitalization in pediatric and adolescent patients. These findings are considered a class effect of LABA monotherapy. When LABA are used in fixed-dose combination with ICS (such as AirDuo Digihaler), data from large clinical trials do not show a significant increase in the risk of serious asthma-related events (hospitalizations, intubations, death) compared with ICS alone

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Inviting Patients (1 of 5)

After you've logged into your specific program, you'll be directed to your Patient List:

Please Note that you can switch between "Patient List" and "Team List" from the left hand navigation menu. To add Patients, ensure "Patient List" is selected. Only Admins and Super Admins will be able to view the Team List and Program Settings.

The screenshot shows a web interface for a "Demo Program" (example-email@test.com). The main heading is "Patient List" with an "Invite Patient" button. Below the heading, it says "125 Patients" and includes a search bar and "Filter" and "Export Data" buttons. A table displays patient data:

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|------------------|------------|--------------------------------|--------|----------------|---------------------------------|--------|----------------|-----------------------------------|--------|----------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1958 | 40 | 38% | 30 minutes ago | 22 | - | - | - | - | - |
| Demo, Patient 2 | 04/26/1964 | 17 | 100% | 5 hours ago | - | - | - | 44 | 100% | - |
| Demo, Patient 3 | 03/22/1966 | 12 | 83% | 23 hours ago | - | - | 2 months ago | - | - | - |
| Demo, Patient 4 | 11/14/1978 | 5 | 60% | 20 minutes ago | 43 | - | 20 minutes ago | - | - | - |
| Demo, Patient 5 | 02/06/1947 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 6 | 01/11/1984 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 7 | 09/05/1957 | 3 | 0% | 3 days ago | - | - | - | - | 64% | 3 days ago |
| Demo, Patient 8 | 10/24/1999 | 0 | - | 3 hours ago | 10 | 30% | 3 hours ago | 10 | 0% | 2 days ago |
| Demo, Patient 9 | 04/16/1950 | 0 | - | 3 hours ago | 10 | 30% | 4 days ago | - | - | - |
| Demo, Patient 10 | 02/04/1948 | 0 | - | 1 month ago | 10 | 30% | 5 days ago | 0 | - | 1 month ago |
| Demo, Patient 11 | 09/21/1997 | 0 | - | 3 days ago | - | - | - | 0 | - | 2 months ago |

Please Note that the Dashboard will only display data for inhalers designated for your program and that have been registered by your patient. For instance, if your program has designated ProAir® Digihaler® only, you will only see data for that inhaler.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Deterioration of Asthma:** Need for more doses of ProAir Digihaler than usual may be a marker of acute or chronic deterioration of asthma and requires reevaluation of treatment, such as possible need for anti-inflammatory treatment, e.g., corticosteroids

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Deterioration of Disease and Acute Episodes:** AirDuo Digihaler should not be initiated in patients during rapidly deteriorating or potentially life-threatening episodes of asthma. ArmonAir Digihaler and AirDuo Digihaler are not indicated for the relief of acute bronchospasm. An inhaled, short-acting beta2-agonist, not ArmonAir Digihaler or AirDuo Digihaler, should be used to relieve acute symptoms such as shortness of breath

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Inviting Patients (2 of 5)

The Patient List is where you can begin to invite participating patients.

Select "Invite Patient" on the top right side of the screen

An invitation code will be generated by the Dashboard. Share the code with your patient via the communication of your choice.

Please Note that the invitation code is not tied specifically to a particular patient, however, it can only be used by one patient.

Invite patient

Provide the invitation code below to the patient. The invitation code must be used within 7 days and can only be used once.

Invitation Code: 63dd1f925f

Once they enter it in their Digihaler App, the patient will appear in the list of patients and any newly generated data will be available to you.

Copy Text

EXPORT PATIENT DATA

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Use of Anti-Inflammatory Agents:** ProAir Digihaler alone may not be adequate to control asthma in many patients. Early consideration should be given to adding anti-inflammatory agents, e.g., corticosteroids


IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Avoid Excessive Use and Avoid Use with Other Long acting Beta2-Agonists:** AirDuo Digihaler should not be used more often than recommended, at higher doses than recommended, or in conjunction with other medicines containing LABA, as an overdose may result. Clinically significant cardiovascular effects and fatalities have been reported in association with excessive use of inhaled sympathomimetic drugs.
Patients using AirDuo Digihaler should not use another medicine containing a LABA (e.g., salmeterol, formoterol fumarate, arformoterol tartrate, indacaterol) for any reason

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Inviting Patients (3 of 5)

The Patient List is where you can begin to invite participating patients.

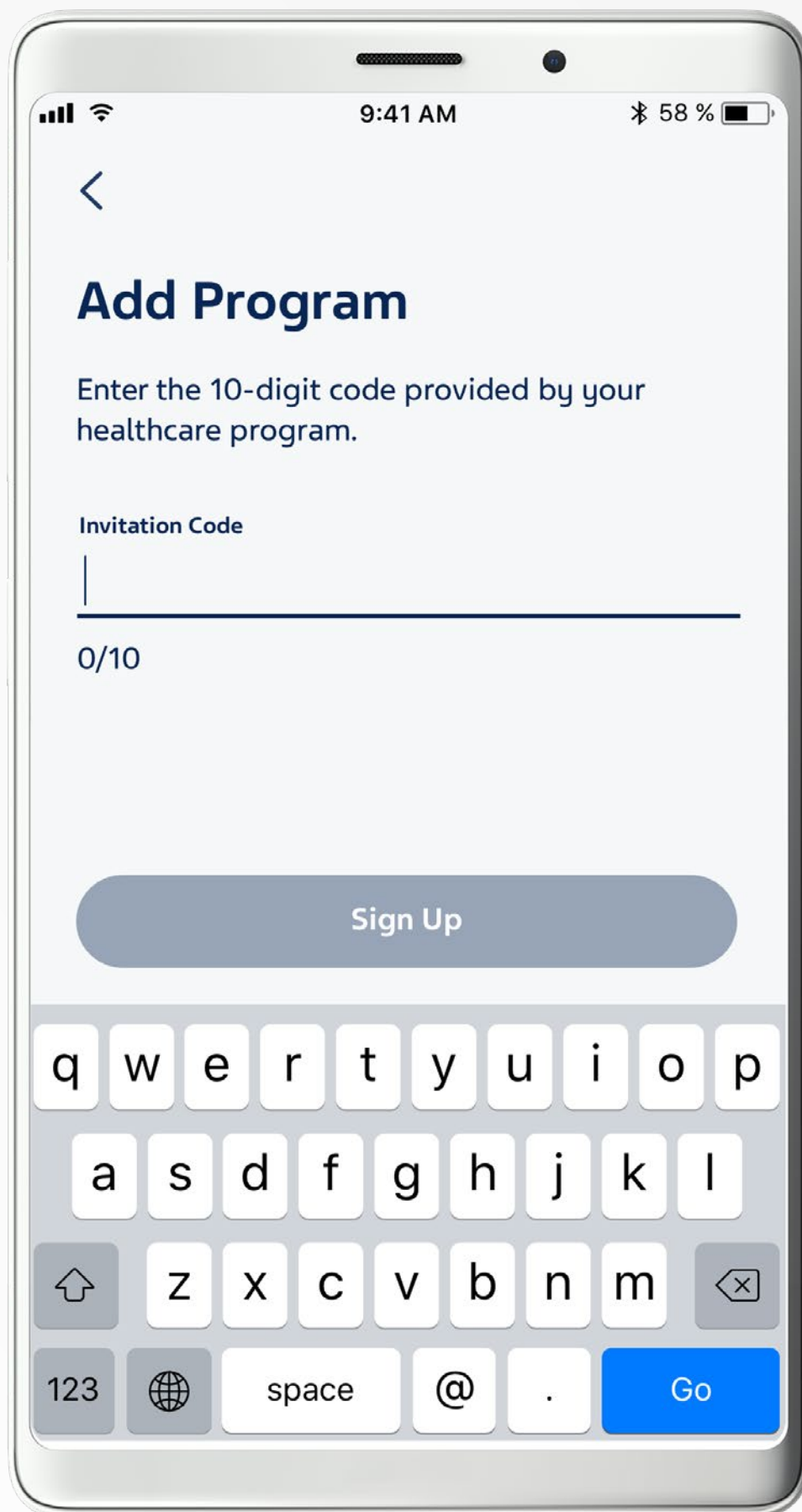

Patients can then enter their code into the “Healthcare Program Screen” in their Digihaler® Mobile App to join, found by tapping the Profile button in the App

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Cardiovascular Effects:** ProAir Digihaler, like other beta-adrenergic agonists, can produce clinically significant cardiovascular effects in some patients, as measured by heart rate, blood pressure, and/or symptoms. If such effects occur, the drug may need to be discontinued. ProAir Digihaler, like all sympathomimetic amines, should be used with caution in patients with cardiovascular disorders, especially coronary insufficiency, cardiac arrhythmias, and hypertension


IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Oropharyngeal Candidiasis** has occurred in patients treated with ArmonAir Digihaler or AirDuo Digihaler. Advise patients to rinse the mouth with water without swallowing following inhalation

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Inviting Patients (4 of 5)

The Patient List is where you can begin to invite participating patients.

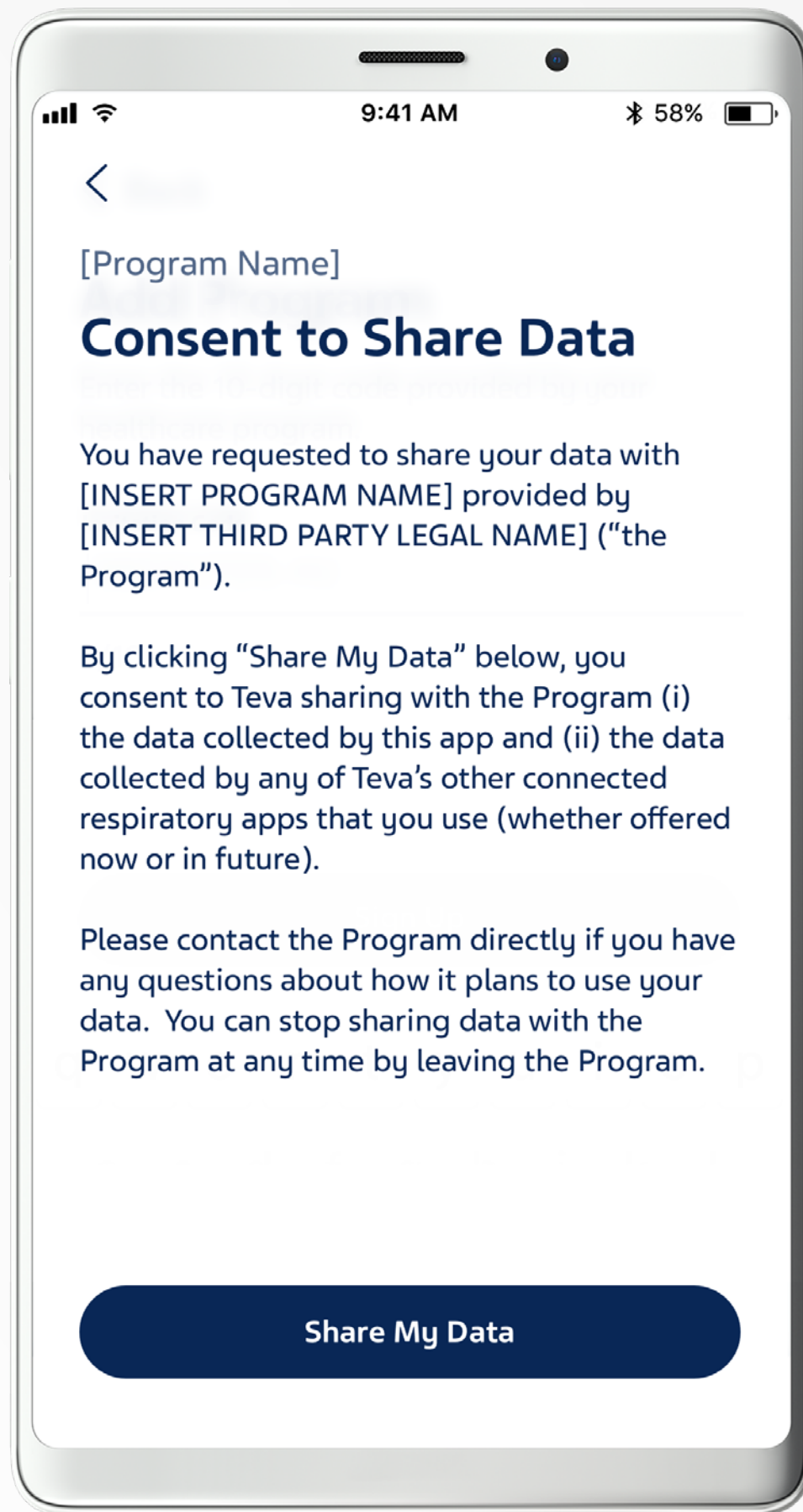

Once the code is validated in the app, the patient will be prompted to consent to share their data

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Do Not Exceed Recommended Dose:** Fatalities have been reported in association with excessive use of inhaled sympathomimetic drugs in patients with asthma

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Immunosuppression and Risks of Infections:** Patients who use corticosteroids, such as found in AirDuo Digihaler and ArmonAir Digihaler are at risk for potential worsening of existing tuberculosis; fungal, bacterial, viral, or parasitic infections; or ocular herpes simplex. A more serious or even fatal course of chickenpox or measles may occur in susceptible patients. Use with caution in patients with the above because of the potential for worsening of these infections

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Inviting Patients (5 of 5)

The Patient List is where you can begin to invite participating patients.

After the patient agrees, you will see their name populated in the Patient List. By default, patients are sorted by the last name.

The screenshot shows a web interface for a 'Demo Program'. The main content area is titled 'Patient List' and displays a table of 125 patients. The table has columns for 'Last, First', 'DOB', and usage statistics for three inhalers: ProAir Digihaler (Last 7 Days), AirDuo Digihaler (Last 30 Days), and ArmonAir Digihaler (Last 30 Days). Each inhaler column includes sub-columns for 'Events', '% Good', and 'All Connected' with a timestamp. A sidebar on the left contains navigation options like 'Patient List', 'Team List', and 'Program Settings'. A top navigation bar includes 'PRESCRIBING INFORMATION' and 'IMPORTANT SAFETY INFORMATION'.

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|------------------|------------|--------------------------------|--------|----------------|---------------------------------|--------|----------------|-----------------------------------|--------|----------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1958 | 40 | 38% | 30 minutes ago | 22 | - | - | - | - | - |
| Demo, Patient 2 | 04/26/1964 | 17 | 100% | 5 hours ago | - | - | - | 44 | 100% | - |
| Demo, Patient 3 | 03/22/1966 | 12 | 83% | 23 hours ago | - | - | 2 months ago | - | - | - |
| Demo, Patient 4 | 11/14/1978 | 5 | 60% | 20 minutes ago | 43 | - | 20 minutes ago | - | - | - |
| Demo, Patient 5 | 02/06/1947 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 6 | 01/11/1984 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 7 | 09/05/1957 | 3 | 0% | 3 days ago | - | - | - | - | 64% | 3 days ago |
| Demo, Patient 8 | 10/24/1999 | 0 | - | 3 hours ago | 10 | 30% | 3 hours ago | 10 | 0% | 2 days ago |
| Demo, Patient 9 | 04/16/1950 | 0 | - | 3 hours ago | 10 | 30% | 4 days ago | - | - | - |
| Demo, Patient 10 | 02/04/1948 | 0 | - | 1 month ago | 10 | 30% | 5 days ago | 0 | - | 1 month ago |
| Demo, Patient 11 | 09/21/1997 | 0 | - | 3 days ago | - | - | - | 0 | - | 2 months ago |

Please Note that the Dashboard will only display data for inhalers designated for your program and that have been registered by your patient. For instance, if your program has designated ProAir® Digihaler® only, you will only see data for that inhaler.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Hypersensitivity Reactions including Anaphylaxis:** Immediate hypersensitivity reactions may occur after administration of albuterol sulfate, as demonstrated by rare cases of urticaria, angioedema, rash, bronchospasm, anaphylaxis, and oropharyngeal edema. Hypersensitivity reactions including anaphylaxis, angioedema, pruritus, and rash have been reported with the use of therapies containing lactose, an inactive ingredient in ProAir Digihaler.

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Transferring Patients from Systemic Corticosteroid Therapy:** Particular care is needed for patients who have been transferred from systemically active corticosteroids to ICS because deaths due to adrenal insufficiency have occurred in patients with asthma during and after transfer from systemic corticosteroids to less systemically available ICS. Taper patients slowly from systemic corticosteroids if transferring to ArmonAir Digihaler or AirDuo Digihaler

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Section 3: Navigating The Patient List

Viewing the Patient List (1 of 7)

Your Patient List is where you can access inhaler event data for each of your patients, organized by the medication they are prescribed. **You can search for a patient by their name, and sort the list by name, date of birth, number of events, percentage of good inhalations, or time since all connected.** The list provides an overarching view of patient data.

The screenshot shows a web interface for a 'Demo Program'. The main content area is titled 'Patient List' and displays a table of 125 patients. The table has columns for 'Last, First', 'DOB', and three inhaler categories: 'ProAir Digihaler (Last 7 Days)', 'AirDuo Digihaler (Last 30 Days)', and 'ArmonAir Digihaler (Last 30 Days)'. Each inhaler category has sub-columns for 'Events', '% Good', and 'All Connected'. The interface includes a search bar, a filter button, an 'Export Data' button, and a pagination control showing '1 - 100 of 125'. A sidebar on the left contains navigation options like 'Patient List', 'Team List', and 'Program Settings'.

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|------------------|------------|--------------------------------|--------|----------------|---------------------------------|--------|----------------|-----------------------------------|--------|----------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1958 | 40 | 38% | 30 minutes ago | 22 | - | - | - | - | - |
| Demo, Patient 2 | 04/26/1964 | 17 | 100% | 5 hours ago | - | - | - | 44 | 100% | - |
| Demo, Patient 3 | 03/22/1966 | 12 | 83% | 23 hours ago | - | - | 2 months ago | - | - | - |
| Demo, Patient 4 | 11/14/1978 | 5 | 60% | 20 minutes ago | 43 | - | 20 minutes ago | - | - | - |
| Demo, Patient 5 | 02/06/1947 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 6 | 01/11/1984 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 7 | 09/05/1957 | 3 | 0% | 3 days ago | - | - | - | - | 64% | 3 days ago |
| Demo, Patient 8 | 10/24/1999 | 0 | - | 3 hours ago | 10 | 30% | 3 hours ago | 10 | 0% | 2 days ago |
| Demo, Patient 9 | 04/16/1950 | 0 | - | 3 hours ago | 10 | 30% | 4 days ago | - | - | - |
| Demo, Patient 10 | 02/04/1948 | 0 | - | 1 month ago | 10 | 30% | 5 days ago | 0 | - | 1 month ago |
| Demo, Patient 11 | 09/21/1997 | 0 | - | 3 days ago | - | - | - | 0 | - | 2 months ago |

Please Note that the Dashboard will only display data for inhalers designated for your program and that have been registered by your patient. For instance, if your program has designated ProAir® Digihaler® only, you will only see data for that inhaler.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Coexisting Conditions:** ProAir Digihaler, like all sympathomimetic amines, should be used with caution in patients with convulsive disorders, hyperthyroidism, or diabetes mellitus; and in patients who are unusually responsive to sympathomimetic amines.

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Hypercorticism and Adrenal Suppression** may occur with high doses of ICS, including fluticasone propionate, or at the recommended dose in susceptible individuals. If such changes occur, discontinue ArmonAir Digihaler or AirDuo Digihaler slowly

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Viewing the Patient List (2 of 7)

The Patient List contains:

Patient info
First name, last name, and date of birth

The screenshot shows a dashboard for a 'Demo Program' with an email 'example-email@test.com'. The main content area is titled 'Patient List' and shows '125 Patients'. There is a search bar 'Search list by name' and buttons for 'Filter' and 'Export Data'. The table below lists 11 patients with their details and usage data for three inhalers: ProAir Digihaler (Last 7 Days), AirDuo Digihaler (Last 30 Days), and ArmonAir Digihaler (Last 30 Days). The table columns are: Last, First; DOB; ProAir Digihaler (Last 7 Days) with sub-columns for Events, % Good, and All Connected; AirDuo Digihaler (Last 30 Days) with sub-columns for Events, % Good, and All Connected; and ArmonAir Digihaler (Last 30 Days) with sub-columns for Events, % Good, and All Connected.

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|------------------|------------|--------------------------------|--------|----------------|---------------------------------|--------|----------------|-----------------------------------|--------|----------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1958 | 40 | 38% | 30 minutes ago | 22 | - | - | - | - | - |
| Demo, Patient 2 | 04/26/1964 | 17 | 100% | 5 hours ago | - | - | - | 44 | 100% | - |
| Demo, Patient 3 | 03/22/1966 | 12 | 83% | 23 hours ago | - | - | 2 months ago | - | - | - |
| Demo, Patient 4 | 11/14/1978 | 5 | 60% | 20 minutes ago | 43 | - | 20 minutes ago | - | - | - |
| Demo, Patient 5 | 02/06/1947 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 6 | 01/11/1984 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 7 | 09/05/1957 | 3 | 0% | 3 days ago | - | - | - | - | 64% | 3 days ago |
| Demo, Patient 8 | 10/24/1999 | 0 | - | 3 hours ago | 10 | 30% | 3 hours ago | 10 | 0% | 2 days ago |
| Demo, Patient 9 | 04/16/1950 | 0 | - | 3 hours ago | 10 | 30% | 4 days ago | - | - | - |
| Demo, Patient 10 | 02/04/1948 | 0 | - | 1 month ago | 10 | 30% | 5 days ago | 0 | - | 1 month ago |
| Demo, Patient 11 | 09/21/1997 | 0 | - | 3 days ago | - | - | - | 0 | - | 2 months ago |

Please Note that the Dashboard will only display data for inhalers designated for your program and that have been registered by your patient. For instance, if your program has designated ProAir® Digihaler® only, you will only see data for that inhaler.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Hypokalemia:** As with other beta-agonists, ProAir Digihaler may produce significant hypokalemia in some patients. The decrease is usually transient, not requiring supplementation

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Drug Interactions with Strong Cytochrome P450 3A4 Inhibitors:** The use of strong cytochrome P450 3A4 (CYP3A4) inhibitors (e.g., ritonavir, ketoconazole) with ArmonAir Digihaler or AirDuo Digihaler is not recommended because increased systemic corticosteroid adverse effects may occur; increased cardiovascular adverse effects may also occur with AirDuo Digihaler

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Viewing the Patient List (3 of 7)

The Patient List contains:

Program inhaler(s)
A list of ProAir® Digihalers®, AirDuo® Digihalers®, and/or ArmonAir® Digihalers® that have been designated for the program

The screenshot shows the 'Patient List' interface. At the top, there are logos for ProAir Digihaler (albuterol sulfate), AirDuo Digihaler (fluticasone propionate and salmeterol), and ArmonAir Digihaler (fluticasone propionate). Below the logos, the title 'Patient List' is displayed. A search bar is present with the text '125 Patients' and 'Search list by name'. There are also 'Filter' and 'Export Data' buttons. A table lists 11 patients with columns for 'Last, First', 'DOB', and usage statistics for three inhaler types: ProAir Digihaler (Last 7 Days), AirDuo Digihaler (Last 30 Days), and ArmonAir Digihaler (Last 30 Days). The table includes columns for 'Events', '% Good', and 'All Connected' for each inhaler type. An 'Invite Patient' button is located in the top right corner.

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|------------------|------------|--------------------------------|--------|----------------|---------------------------------|--------|----------------|-----------------------------------|--------|----------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1958 | 40 | 38% | 30 minutes ago | 22 | - | - | - | - | - |
| Demo, Patient 2 | 04/26/1964 | 17 | 100% | 5 hours ago | - | - | - | 44 | 100% | - |
| Demo, Patient 3 | 03/22/1966 | 12 | 83% | 23 hours ago | - | - | 2 months ago | - | - | - |
| Demo, Patient 4 | 11/14/1978 | 5 | 60% | 20 minutes ago | 43 | - | 20 minutes ago | - | - | - |
| Demo, Patient 5 | 02/06/1947 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 6 | 01/11/1984 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 7 | 09/05/1957 | 3 | 0% | 3 days ago | - | - | - | - | 64% | 3 days ago |
| Demo, Patient 8 | 10/24/1999 | 0 | - | 3 hours ago | 10 | 30% | 3 hours ago | 10 | 0% | 2 days ago |
| Demo, Patient 9 | 04/16/1950 | 0 | - | 3 hours ago | 10 | 30% | 4 days ago | - | - | - |
| Demo, Patient 10 | 02/04/1948 | 0 | - | 1 month ago | 10 | 30% | 5 days ago | 0 | - | 1 month ago |
| Demo, Patient 11 | 09/21/1997 | 0 | - | 3 days ago | - | - | - | 0 | - | 2 months ago |

Please Note that the Dashboard will only display data for inhalers designated for your program and that have been registered by your patient. For instance, if your program has designated ProAir Digihaler only, you will only see data for that inhaler.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Most common adverse reactions** ($\geq 1\%$ and $>$ placebo) are back pain, pain, gastroenteritis viral, sinus headache, urinary tract infection, nasopharyngitis, oropharyngeal pain and vomiting

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Paradoxical Bronchospasm and Upper Airway Symptoms:** Paradoxical bronchospasm may occur. If bronchospasm occurs treat immediately with an inhaled, short-acting bronchodilator discontinue AirDuo Digihaler or ArmonAir Digihaler and institute alternative therapy

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Viewing the Patient List (4 of 7)

The Patient List contains:

Digihaler® Events

Indicates the number of Digihaler Events (i.e. each time the inhaler is used or the cap is opened) that have occurred

The screenshot shows a web interface for a patient list. At the top, there are logos for ProAir Digihaler (albuterol sulfate inhalation powder), AirDuo Digihaler (fluticasone propionate and salmeterol inhalation powder), and ArmonAir Digihaler (fluticasone propionate inhalation powder). Navigation links include 'PRESCRIBING INFORMATION' and 'IMPORTANT SAFETY INFORMATION'. A sidebar on the left contains 'Demo Program', 'Patient List', 'Team List', and 'Program Settings'. The main content area shows '125 Patients' with a search bar and 'Filter' and 'Export Data' buttons. A table lists 11 patients with columns for 'Last, First', 'DOB', and usage statistics for three inhaler types: ProAir Digihaler (Last 7 Days), AirDuo Digihaler (Last 30 Days), and ArmonAir Digihaler (Last 30 Days). Each inhaler column includes 'Events', '% Good', and 'All Connected' sub-columns.

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|------------------|------------|--------------------------------|--------|----------------|---------------------------------|--------|----------------|-----------------------------------|--------|----------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1958 | 40 | 38% | 30 minutes ago | 22 | - | - | - | - | - |
| Demo, Patient 2 | 04/26/1964 | 17 | 100% | 5 hours ago | - | - | - | 44 | 100% | - |
| Demo, Patient 3 | 03/22/1966 | 12 | 83% | 23 hours ago | - | - | 2 months ago | - | - | - |
| Demo, Patient 4 | 11/14/1978 | 5 | 60% | 20 minutes ago | 43 | - | 20 minutes ago | - | - | - |
| Demo, Patient 5 | 02/06/1947 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 6 | 01/11/1984 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 7 | 09/05/1957 | 3 | 0% | 3 days ago | - | - | - | - | 64% | 3 days ago |
| Demo, Patient 8 | 10/24/1999 | 0 | - | 3 hours ago | 10 | 30% | 3 hours ago | 10 | 0% | 2 days ago |
| Demo, Patient 9 | 04/16/1950 | 0 | - | 3 hours ago | 10 | 30% | 4 days ago | - | - | - |
| Demo, Patient 10 | 02/04/1948 | 0 | - | 1 month ago | 10 | 30% | 5 days ago | 0 | - | 1 month ago |
| Demo, Patient 11 | 09/21/1997 | 0 | - | 3 days ago | - | - | - | 0 | - | 2 months ago |

Please Note that the Dashboard will only display data for inhalers designated for your program and that have been registered by your patient. For instance, if your program has designated ProAir® Digihaler® only, you will only see data for that inhaler.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** Other short-acting sympathomimetic bronchodilators should not be used concomitantly with ProAir Digihaler

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Hypersensitivity Reactions, Including Anaphylaxis:** Immediate hypersensitivity reactions (e.g., urticaria, angioedema, rash, bronchospasm, hypotension), including anaphylaxis, may occur after administration of ArmonAir Digihaler or AirDuo Digihaler. Discontinue ArmonAir Digihaler or AirDuo Digihaler if such reactions occur

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Viewing the Patient List (5 of 7)

The Patient List contains:

% Good

The percentage of events that were categorized as “Good Inhalations” by the Digihaler® Mobile App

Each event is categorized as: good, fair, low or no inhalation

(>45 L/min; 30-45 L/min; <30 L/min respectively), exhalation, or possible air vent block

The screenshot shows a dashboard for a "Demo Program" with an email "example-email@test.com". The main section is titled "Patient List" and shows "125 Patients". There is a search bar "Search list by name" and buttons for "Filter" and "Export Data". The table below lists 11 patients with their details and inhaler usage data.

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|------------------|------------|--------------------------------|--------|----------------|---------------------------------|--------|----------------|-----------------------------------|--------|----------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1958 | 40 | 38% | 30 minutes ago | 22 | - | - | - | - | - |
| Demo, Patient 2 | 04/26/1964 | 17 | 100% | 5 hours ago | - | - | - | 44 | 100% | - |
| Demo, Patient 3 | 03/22/1966 | 12 | 83% | 23 hours ago | - | - | 2 months ago | - | - | - |
| Demo, Patient 4 | 11/14/1978 | 5 | 60% | 20 minutes ago | 43 | - | 20 minutes ago | - | - | - |
| Demo, Patient 5 | 02/06/1947 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 6 | 01/11/1984 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 7 | 09/05/1957 | 3 | 0% | 3 days ago | - | - | - | - | 64% | 3 days ago |
| Demo, Patient 8 | 10/24/1999 | 0 | - | 3 hours ago | 10 | 30% | 3 hours ago | 10 | 0% | 2 days ago |
| Demo, Patient 9 | 04/16/1950 | 0 | - | 3 hours ago | 10 | 30% | 4 days ago | - | - | - |
| Demo, Patient 10 | 02/04/1948 | 0 | - | 1 month ago | 10 | 30% | 5 days ago | 0 | - | 1 month ago |
| Demo, Patient 11 | 09/21/1997 | 0 | - | 3 days ago | - | - | - | 0 | - | 2 months ago |

Please Note that the Dashboard will only display data for inhalers designated for your program and that have been registered by your patient. For instance, if your program has designated ProAir® Digihaler® only, you will only see data for that inhaler.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** (continued)
 - » **Beta-Blockers:** Beta-adrenergic-receptor blocking agents not only block the pulmonary effect of beta-agonists, such as ProAir Digihaler, but may produce severe bronchospasm in asthmatic patients. Therefore, patients with asthma should not normally be treated with beta-blockers

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Cardiovascular and Central Nervous System Effects:** The salmeterol component of AirDuo Digihaler, can be associated with excessive beta-adrenergic stimulation which could present as the following symptoms: seizures, angina, hypertension or hypotension, tachycardia with rates up to 200 beats/min, arrhythmias, nervousness, headache, tremor, palpitation, nausea, dizziness, fatigue, malaise, and insomnia. Use with caution in patients with cardiac arrhythmias, hypertension, coronary insufficiency. Drug may need to be discontinued in certain patients.

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Viewing the Patient List (6 of 7)

The Patient List contains:

All Connected

Lists the last known time that the patient's ProAir® Digihaler®, AirDuo® Digihaler®, and/or ArmonAir® Digihaler® were connected to the app
Please Note that this information will update when the inhaler is connected to the Digihaler Mobile App and the patient's Bluetooth® Wireless Technology is turned on and within range

The screenshot shows a dashboard for a 'Demo Program' with a 'Patient List' section. The list contains 125 patients. A search bar and filter options are available. The table below shows the first 11 patients.

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|------------------|------------|--------------------------------|--------|----------------|---------------------------------|--------|----------------|-----------------------------------|--------|----------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1958 | 40 | 38% | 30 minutes ago | 22 | - | - | - | - | - |
| Demo, Patient 2 | 04/26/1964 | 17 | 100% | 5 hours ago | - | - | - | 44 | 100% | |
| Demo, Patient 3 | 03/22/1966 | 12 | 83% | 23 hours ago | - | - | 2 months ago | - | - | - |
| Demo, Patient 4 | 11/14/1978 | 5 | 60% | 20 minutes ago | 43 | - | 20 minutes ago | - | - | - |
| Demo, Patient 5 | 02/06/1947 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 6 | 01/11/1984 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 7 | 09/05/1957 | 3 | 0% | 3 days ago | - | - | - | - | 64% | 3 days ago |
| Demo, Patient 8 | 10/24/1999 | 0 | - | 3 hours ago | 10 | 30% | 3 hours ago | 10 | 0% | 2 days ago |
| Demo, Patient 9 | 04/16/1950 | 0 | - | 3 hours ago | 10 | 30% | 4 days ago | - | - | - |
| Demo, Patient 10 | 02/04/1948 | 0 | - | 1 month ago | 10 | 30% | 5 days ago | 0 | - | 1 month ago |
| Demo, Patient 11 | 09/21/1997 | 0 | - | 3 days ago | - | - | - | 0 | - | 2 months ago |

Please Note that the Dashboard will only display data for inhalers designated for your program and that have been registered by your patient. For instance, if your program has designated ProAir Digihaler only, you will only see data for that inhaler.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** (continued)
 - » **Diuretics:** Caution is advised in the coadministration of beta-agonists with non-potassium sparing diuretics (such as loop or thiazide diuretics). Consider monitoring potassium levels

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Reduction in Bone Mineral Density (BMD):** Decreases in BMD have been observed with long-term administration of products containing ICS. Patients with major risk factors for decreased bone mineral content, such as prolonged immobilization, family history of osteoporosis, or chronic use of drugs that can reduce bone mass (e.g., anticonvulsants, oral corticosteroids) should be monitored and treated with established standards of care when using ArmonAir Digihaler or AirDuo Digihaler

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Viewing the Patient List (7 of 7)

The Patient List contains:

Please Note that hovering over the “?” icon will display helpful definitions and lead to further support pages

The screenshot shows the 'Patient List' interface. At the top, there are logos for ProAir Digihaler (albuterol sulfate inhalation powder), AirDuo Digihaler (fluticasone propionate and salmeterol inhalation powder), and ArmonAir Digihaler (fluticasone propionate inhalation powder). Below the logos, there are links for 'PRESCRIBING INFORMATION' and 'IMPORTANT SAFETY INFORMATION'. A navigation sidebar on the left includes 'Demo Program', 'Patient List', 'Team List', and 'Program Settings'. The main content area shows '125 Patients' with a search bar and 'Filter' and 'Export Data' buttons. A table lists 11 patients with columns for 'Last, First', 'DOB', and data for three inhalers: ProAir Digihaler (Last 7 Days), AirDuo Digihaler (Last 30 Days), and ArmonAir Digihaler (Last 30 Days). A tooltip is displayed over a question mark icon in the ProAir column, providing definitions for 'Events', '% Good', and 'All Connected'.

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | AirDuo Digihaler (Last 30 Days) | ArmonAir Digihaler (Last 30 Days) |
|------------------|------------|--------------------------------|---------------------------------|-----------------------------------|
| Demo, Patient 1 | 08/06/1958 | | | |
| Demo, Patient 2 | 04/26/1964 | | | |
| Demo, Patient 3 | 03/22/1966 | | | |
| Demo, Patient 4 | 11/14/1978 | | | |
| Demo, Patient 5 | 02/06/1947 | 4 0% 30 minutes ago | - - - | 0 - 15 minutes ago |
| Demo, Patient 6 | 01/11/1984 | 4 0% 30 minutes ago | - - - | 0 - 15 minutes ago |
| Demo, Patient 7 | 09/05/1957 | 3 0% 3 days ago | - - - | - 64% 3 days ago |
| Demo, Patient 8 | 10/24/1999 | 0 - 3 hours ago | 10 30% 3 hours ago | 10 0% 2 days ago |
| Demo, Patient 9 | 04/16/1950 | 0 - 3 hours ago | 10 30% 4 days ago | - - - |
| Demo, Patient 10 | 02/04/1948 | 0 - 1 month ago | 10 30% 5 days ago | 0 - 1 month ago |
| Demo, Patient 11 | 09/21/1997 | 0 - 3 days ago | - - - | 0 - 2 months ago |

Please Note that the Dashboard will only display data for inhalers designated for your program and that have been registered by your patient. For instance, if your program has designated ProAir® Digihaler® only, you will only see data for that inhaler.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** (continued)
 - » **Digoxin:** Carefully evaluate the serum digoxin levels in patients who are currently receiving digoxin and ProAir Digihaler

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Effect on Growth:** ICS may cause a reduction in growth velocity, Patients should be maintained on the lowest dose of inhaled corticosteroid that effectively controls their asthma. Monitor growth of pediatric patients receiving ArmonAir Digihaler and AirDuo Digihaler.

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Filtering the Patient List

By default, the Patient List will display an unfiltered list with All Patients data. When managing larger populations of patients on the Dashboard, it may prove challenging to quickly identify specific patient cohorts. The filtering feature allows Admins and Super Admins to create custom filters that all users of the Dashboard can then leverage to identify patients that match defined criteria ([See Section 8: Program Settings](#)).

Custom filters will filter the list of patients based on how many events patients had that match criteria such as inhaler brand, time of day, or range of dates. Depending on whether the patient is over or under a certain threshold that matches the defined criteria, this will determine whether the patient is displayed on the filtered list or not.

The screenshot shows the 'Patient List' interface. At the top, there are logos for ProAir Digihaler, AirDuo Digihaler, and ArmonAir Digihaler, along with links for 'PRESCRIBING INFORMATION' and 'IMPORTANT SAFETY INFORMATION'. The main header includes 'Patient List' and an 'Invite Patient' button. Below the header, there is a search bar and a 'Filter' button. A table displays patient data with columns for 'Last, First', 'DOB', and three inhaler categories: 'ProAir Digihaler (Last 7 Days)', 'AirDuo Digihaler (Last 30 Days)', and 'ArmonAir Digihaler (Last 30 Days)'. Each inhaler category has sub-columns for 'Events', '% Good', and 'All Connected'. A dropdown menu is open from the 'Filter' button, showing example filter names: 'Decrease in Controller Use', 'Increase in Rescue Use', 'Low Controller Use', and 'Nighttime Rescue Use'.

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|-----------------|------------|--------------------------------|--------|----------------|---------------------------------|--------|----------------|-----------------------------------|--------|---------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1958 | 40 | 38% | 30 minutes ago | 22 | - | - | - | - | - |
| Demo, Patient 2 | 04/26/1964 | 17 | 100% | 5 hours ago | - | - | - | 44 | 100% | - |
| Demo, Patient 3 | 03/22/1966 | 12 | 83% | 23 hours ago | - | - | 2 months ago | - | - | - |
| Demo, Patient 4 | 11/14/1978 | 5 | 60% | 20 minutes ago | 43 | - | 20 minutes ago | - | - | - |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** (continued)
 - » **Monoamine Oxidase Inhibitors or Tricyclic Antidepressants:** ProAir Digihaler should be administered with extreme caution to patients being treated with these agents, or within 2 weeks of discontinuation of these agents, because the action of albuterol on the cardiovascular system may be potentiated. Consider alternative therapy

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Glaucoma and Cataracts:** Long-term use of ICS, including fluticasone propionate, a component of ArmonAir Digihaler and AirDuo Digihaler, may increase the risk for cataracts or glaucoma. Regular eye exams should be considered

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Removing Patients from your Program (1 of 3)

You are able to remove patients from your program via the "Patient Details" page. To do so:

- Go to the "Patient List" page
- Select the patient you want to remove

Click the patient's name on your Patient List to view their "Patient Details" page

ProAir digihaler (albuterol sulfate) inhalation powder **AirDuo digihaler** (fluticasone propionate and salmeterol) inhalation powder **ArmonAir digihaler** (fluticasone propionate) inhalation powder **PRESCRIBING INFORMATION** **IMPORTANT SAFETY INFORMATION**

Patient List Invite Patient

125 Patients Search list by name Filter Export Data 1 - 100 of 125

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|-----------------|------------|--------------------------------|--------|----------------|---------------------------------|--------|----------------|-----------------------------------|--------|----------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1956 | 40 | 38% | 30 minutes ago | 22 | - | - | - | - | - |
| Demo, Patient 2 | 04/26/1964 | 17 | 100% | 5 hours ago | - | - | - | 44 | 100% | - |
| Demo, Patient 3 | 03/22/1966 | 12 | 83% | 23 hours ago | - | - | 2 months ago | - | - | - |
| Demo, Patient 4 | 11/14/1978 | 5 | 60% | 20 minutes ago | 43 | - | 20 minutes ago | - | - | - |
| Demo, Patient 5 | 02/06/1947 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 6 | 01/11/1984 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- Paradoxical Bronchospasm:** ProAir Digihaler can produce paradoxical bronchospasm that may be life-threatening. Discontinue ProAir Digihaler and institute alternative therapy if paradoxical bronchospasm occurs

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)


- Eosinophilic Conditions and Churg-Strauss Syndrome:** Systemic eosinophilic conditions, such as Churg-Strauss syndrome, may occur when using ArmonAir Digihaler or AirDuo Digihaler. Be alert to eosinophilia, vasculitic rash, worsening pulmonary symptoms, cardiac complications, and/or neuropathy

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.


Removing Patients from your Program (2 of 3)

You are able to remove patients from your program via the “Patient Details” page

Click the dots on the top of the details page to show “Remove Patient” button

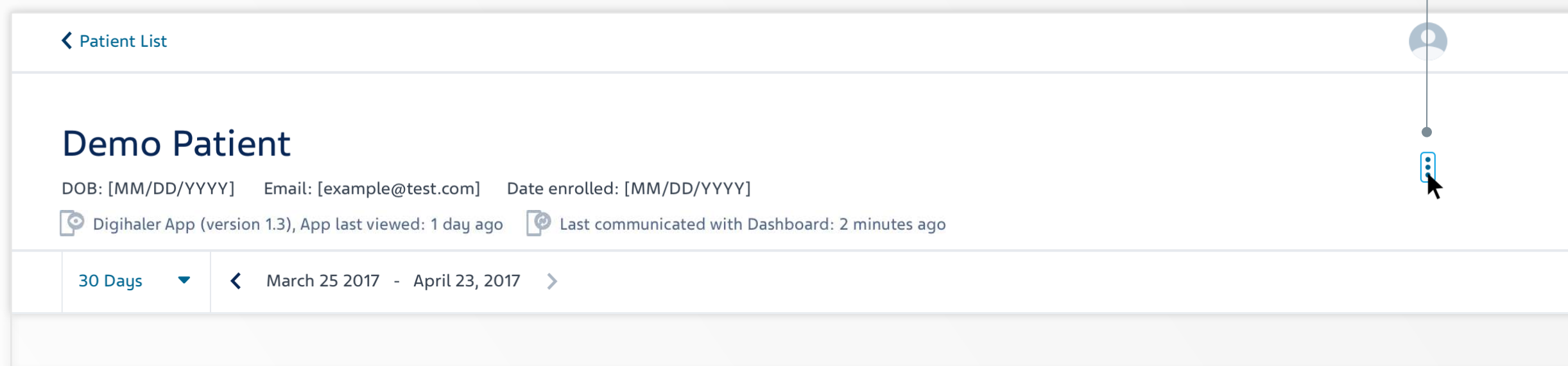

Then click “Remove Patient”

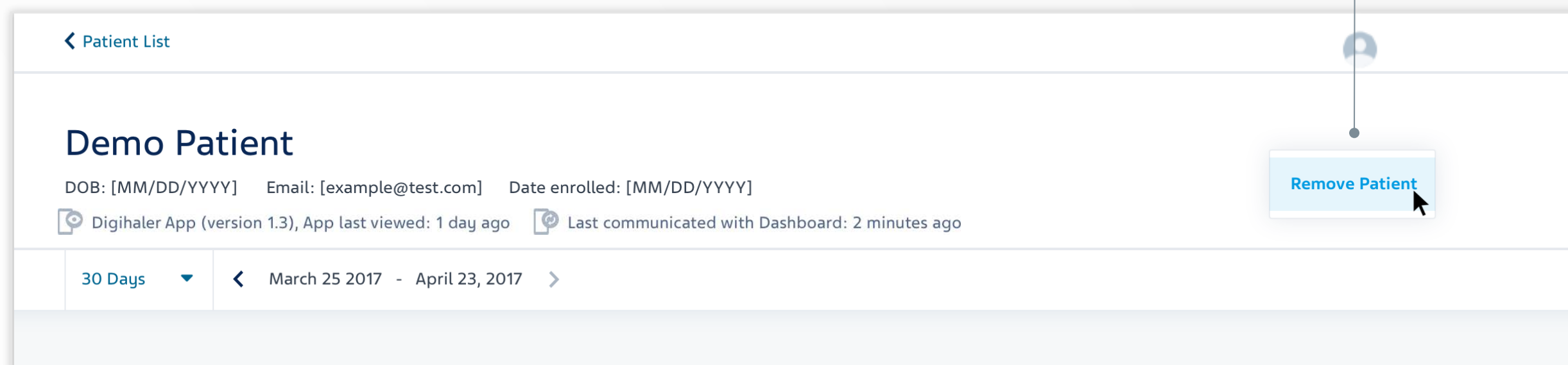

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Deterioration of Asthma:** Need for more doses of ProAir Digihaler than usual may be a marker of acute or chronic deterioration of asthma and requires reevaluation of treatment, such as possible need for anti-inflammatory treatment, e.g., corticosteroids

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)


- **Coexisting Conditions:** Use AirDuo Digihaler with caution in patients with convulsive disorders, thyrotoxicosis, diabetes mellitus, ketoacidosis, and in patients who are unusually responsive to sympathomimetic amines

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Removing Patients from your Program (3 of 3)

When removing a patient, you will be prompted to confirm this action and warned that this operation cannot be undone. After confirmation, their name will disappear from the Patient List. Patients can also leave the program via the Digihaler® Mobile App, and there will be no notification that the patient has done so.

Please Note that the patient's inhaler event data is stored on the inhaler itself, and viewed in the Digihaler Mobile App. If the patient wishes to re-enroll in the program, you simply have to invite them again, and you will still be able to view their previous data after they consent to share.

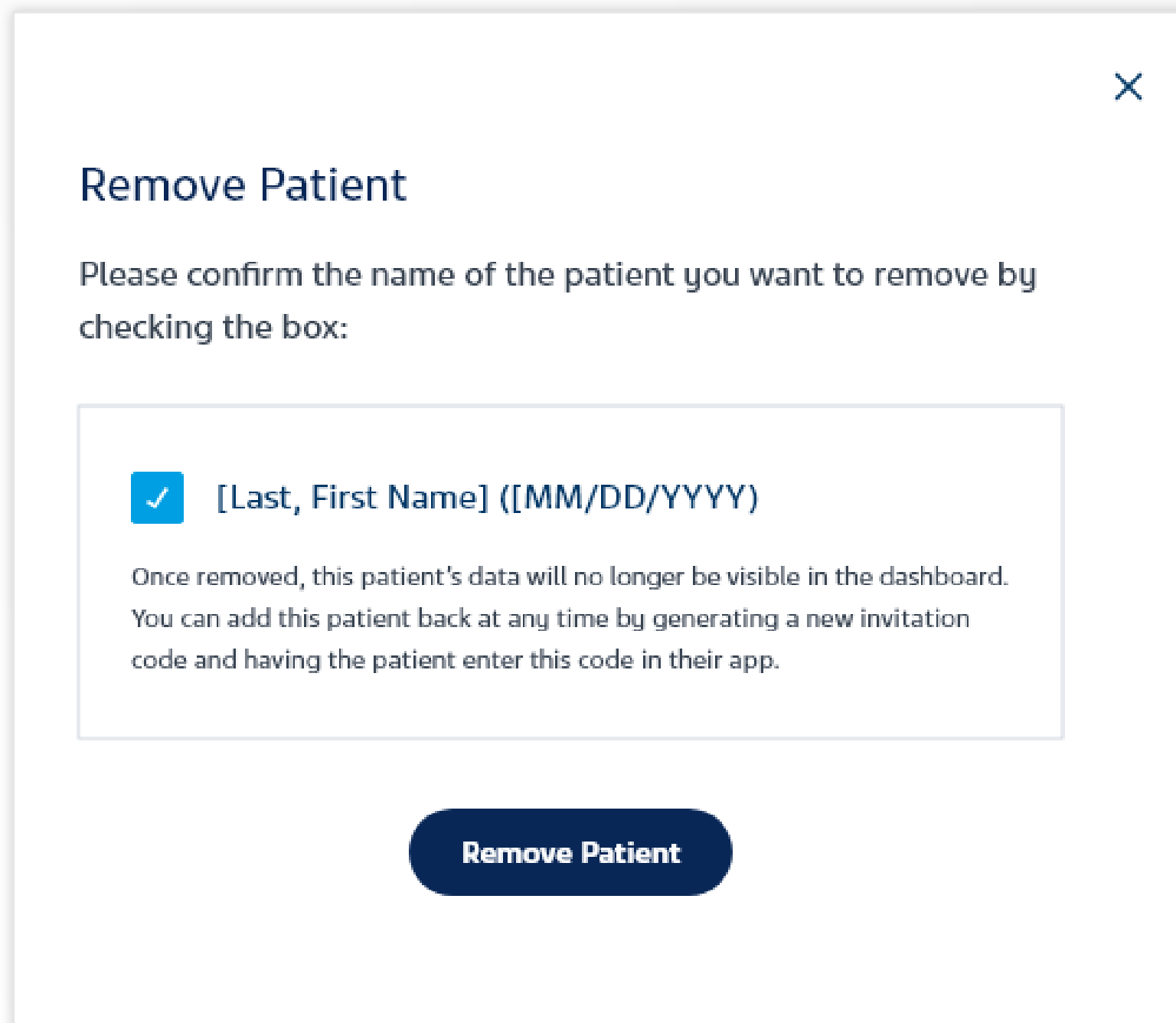

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Use of Anti-Inflammatory Agents:** ProAir Digihaler alone may not be adequate to control asthma in many patients. Early consideration should be given to adding anti-inflammatory agents, e.g., corticosteroids

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Hypokalemia and Hyperglycemia:** Beta-adrenergic agonist medicines may produce significant hypokalemia in some patients, possibly through intracellular shunting, which has the potential to produce adverse cardiovascular effects. Decrease in serum potassium are usually transient, not requiring supplementation. Be alert to hypokalemia and hyperglycemia in patients using AirDuo Digihaler

Please see additional Safety Information throughout this brochure and full Prescribing Information for [ProAir Digihaler](#) [ArmonAir Digihaler](#) and [AirDuo Digihaler](#).

Section 4: Viewing Patient Details

Patient Details (1 of 2)

The Patient Details Screen includes details on inhaler events, such as time of use and inspiratory flow rates. Inhaler use is recorded as an event when a patient opens the cap or inhales. **The Patient Details Screen can help healthcare providers see a broad view of inhaler events, and assess trends or patterns over time.** This information may help inform treatment decisions during consultations with patients.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Cardiovascular Effects:** ProAir Digihaler, like other beta-adrenergic agonists, can produce clinically significant cardiovascular effects in some patients, as measured by heart rate, blood pressure, and/or symptoms. If such effects occur, the drug may need to be discontinued. ProAir Digihaler, like all sympathomimetic amines, should be used with caution in patients with cardiovascular disorders, especially coronary insufficiency, cardiac arrhythmias, and hypertension

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Adverse Reactions with ArmonAir Digihaler:** Most common adverse reactions (greater than or equal to 3%) are: upper respiratory tract infection, nasopharyngitis, oral candidiasis, headache, and cough

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Patient Details (2 of 2)

To view a patient’s detailed information, select that patient’s name from the Patient List. You will be directed to charts and tables related to the patient's inhaler event data, including:


- The Digihaler® Events Chart
- The Recorded Digihaler® Events Table
- Digihaler Connectivity
- Detailed Data Screen

Click a patient's name to view their detailed information

Demo, Patient



| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|------------------|------------|--------------------------------|--------|----------------|---------------------------------|--------|----------------|-----------------------------------|--------|----------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1958 | 40 | 38% | 30 minutes ago | 22 | - | - | - | - | - |
| Demo, Patient 2 | 04/26/1964 | 17 | 100% | 5 hours ago | - | - | - | 44 | 100% | - |
| Demo, Patient 3 | 03/22/1966 | 12 | 83% | 23 hours ago | - | - | 2 months ago | - | - | - |
| Demo, Patient 4 | 11/14/1978 | 5 | 60% | 20 minutes ago | 43 | - | 20 minutes ago | - | - | - |
| Demo, Patient 5 | 02/06/1947 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 6 | 01/11/1984 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 7 | 09/05/1957 | 3 | 0% | 3 days ago | - | - | - | - | 64% | 3 days ago |
| Demo, Patient 8 | 10/24/1999 | 0 | - | 3 hours ago | 10 | 30% | 3 hours ago | 10 | 0% | 2 days ago |
| Demo, Patient 9 | 04/16/1950 | 0 | - | 3 hours ago | 10 | 30% | 4 days ago | - | - | - |
| Demo, Patient 10 | 02/04/1948 | 0 | - | 1 month ago | 10 | 30% | 5 days ago | 0 | - | 1 month ago |
| Demo, Patient 11 | 09/21/1997 | 0 | - | 3 days ago | - | - | - | 0 | - | 2 months ago |

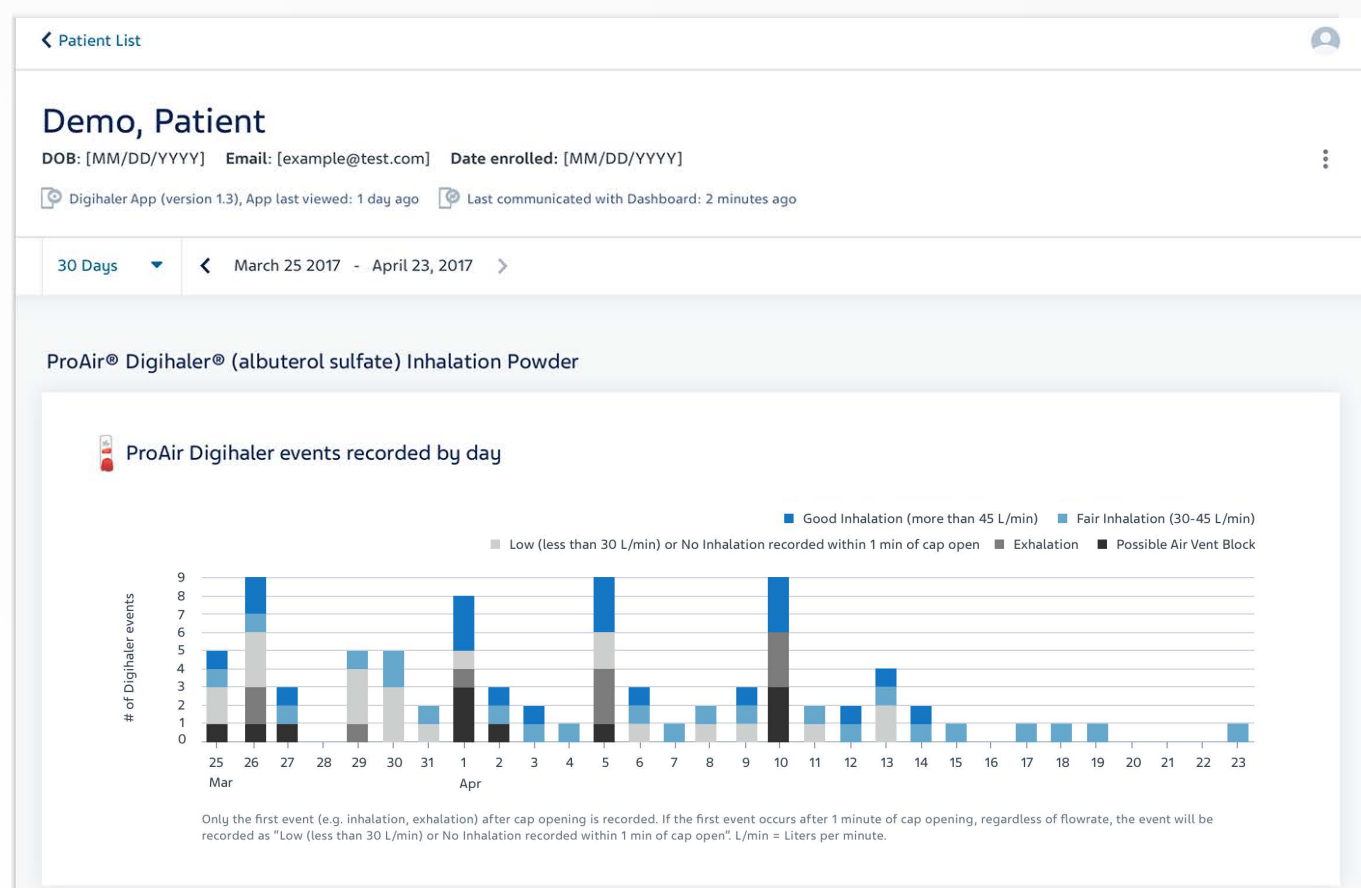

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Do Not Exceed Recommended Dose:** Fatalities have been reported in association with excessive use of inhaled sympathomimetic drugs in patients with asthma

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Adverse Reactions with AirDuo Digihaler:** Most common adverse reactions (greater than or equal to 3%) include nasopharyngitis, oral candidiasis, headache, cough and back pain

Please see additional Safety Information throughout this brochure and full Prescribing Information for [ProAir Digihaler](#), [ArmonAir Digihaler](#) and [AirDuo Digihaler](#).




Understanding the Patient Details Screen (1 of 2)

The Patient Details Screen is comprised of three sections:

1 **The Digihaler® Events Chart**

Consists of Digihaler Events per medication for each registered inhaler in a chart

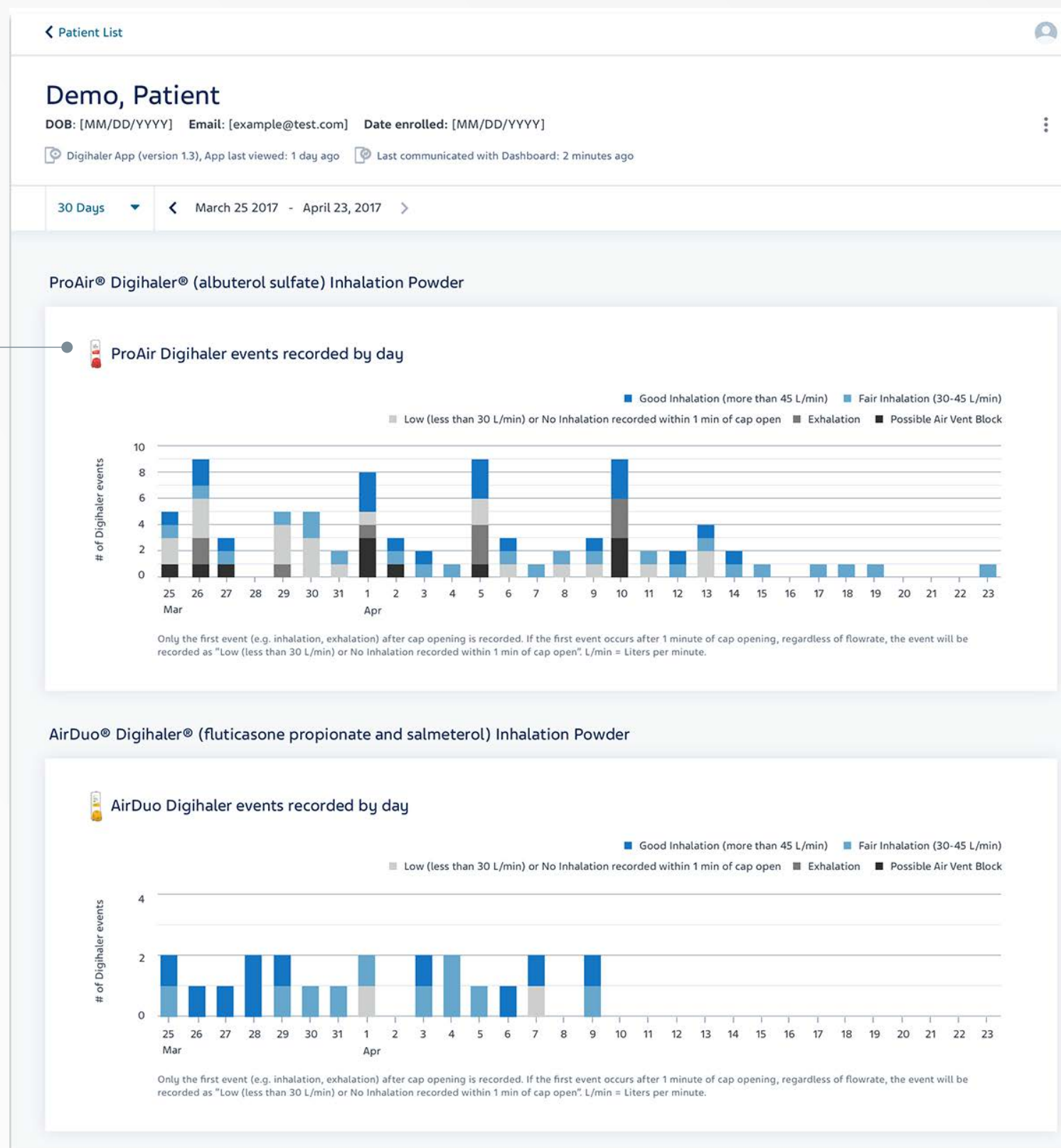

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Hypersensitivity Reactions including Anaphylaxis:** Immediate hypersensitivity reactions may occur after administration of albuterol sulfate, as demonstrated by rare cases of urticaria, angioedema, rash, bronchospasm, anaphylaxis, and oropharyngeal edema. Hypersensitivity reactions including anaphylaxis, angioedema, pruritus, and rash have been reported with the use of therapies containing lactose, an inactive ingredient in ProAir Digihaler.

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Serious Asthma-Related Events – Hospitalizations, Intubations, Death:** Use of a LABA as monotherapy (without an ICS) for asthma is associated with an increased risk of asthma-related death. Available data from controlled clinical trials also suggest that use of LABA as monotherapy increases the risk of asthma-related hospitalization in pediatric and adolescent patients. These findings are considered a class effect of LABA monotherapy. When LABA are used in fixed-dose combination with ICS (such as AirDuo Digihaler), data from large clinical trials do not show a significant increase in the risk of serious asthma-related events (hospitalizations, intubations, death) compared with ICS alone

Please see additional Safety Information throughout this brochure and full Prescribing Information for [ProAir Digihaler](#), [ArmonAir Digihaler](#) and [AirDuo Digihaler](#).

Understanding the Patient Details Screen (2 of 2)

The Patient Details Screen is comprised of three sections:

Recorded Digihaler Events What is a Digihaler Event?

| Date | ProAir Digihaler | | AirDuo Digihaler | | Daily Self-Assessment |
|--------------------|------------------|--------------------|------------------|--------------------|-----------------------|
| | Recorded Events | % Good Inhalations | Recorded Events | % Good Inhalations | |
| 04/17 - 04/23/2017 | 4 | 0% | 0 | - | 5 |
| 04/10 - 04/16/2017 | 20 | 30% | 0 | - | 3 |
| 04/03 - 04/09/2017 | 21 | 24% | 10 | 40% | 0 |
| 03/27 - 04/02/2017 | 26 | 19% | 9 | 55% | 2 |
| 03/25 - 03/26/2017 | 14 | 21% | 3 | 66% | 0 |

Digihaler Connectivity Connectivity Help

| Digihaler Type | Nickname & ID | Added | Removed | Last Connected |
|--|----------------------------|------------|------------|----------------|
| ProAir Digihaler (albuterol sulfate) Inhalation Powder, 117 mcg | Sports ID: XXXXXXXXXXXX | 03/21/2017 | — | 1 hour ago |
| ProAir Digihaler (albuterol sulfate) Inhalation Powder, 117 mcg | Travel ID: XXXXXXXXXXXX | 03/21/2017 | — | a month ago |
| AirDuo Digihaler (fluticasone propionate/salmeterol) Inhalation Powder, 113/14 mcg | April ID: XXXXXXXXXXXX | 04/01/2017 | — | 1 hour ago |
| AirDuo Digihaler (fluticasone propionate/salmeterol) Inhalation Powder, 113/14 mcg | May ID: XXXXXXXXXXXX | 03/01/2017 | 04/01/2017 | 22 days ago |

Version 4.3.0. Copyright © 2022 Teva Respiratory, LLC teva

2 **The Recorded Digihaler® Events Table**
Lists the patient's recorded Digihaler Events, percentage of good inhalations, and Daily Self-Assessments in a table view

3 **Digihaler Connectivity**
All of the patient's registered Digihaler inhalers

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Coexisting Conditions:** ProAir Digihaler, like all sympathomimetic amines, should be used with caution in patients with convulsive disorders, hyperthyroidism, or diabetes mellitus; and in patients who are unusually responsive to sympathomimetic amines.

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)


- **Deterioration of Disease and Acute Episodes:** AirDuo Digihaler should not be initiated in patients during rapidly deteriorating or potentially life-threatening episodes of asthma. ArmonAir Digihaler and AirDuo Digihaler are not indicated for the relief of acute bronchospasm. An inhaled, short-acting beta2-agonist, not ArmonAir Digihaler or AirDuo Digihaler, should be used to relieve acute symptoms such as shortness of breath

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

The Digihaler® Events Chart (1 of 3)

The Digihaler Events chart is the first section of the Patient Details Screen. The chart shows a series of stacked columns that count the number of good inhalations (>45 L/min), fair inhalations (30 to 45 L/min), low/no inhalations (<30 L/min), exhalations and blocked air vent events that occurred for the given day.

The events are categorized as:

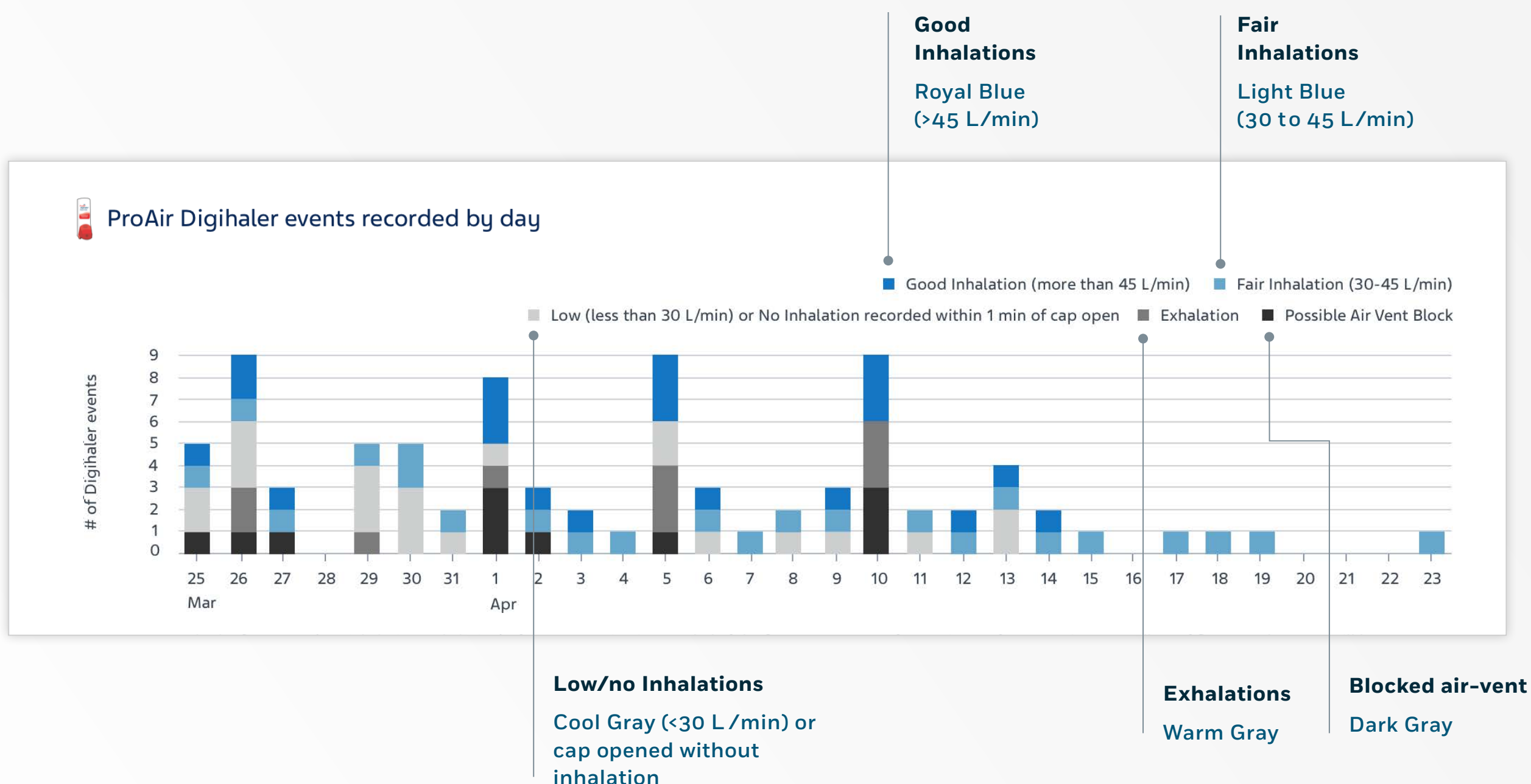

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Hypokalemia:** As with other beta-agonists, ProAir Digihaler may produce significant hypokalemia in some patients. The decrease is usually transient, not requiring supplementation

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Avoid Excessive Use and Avoid Use with Other Long acting Beta2-Agonists:** AirDuo Digihaler should not be used more often than recommended, at higher doses than recommended, or in conjunction with other medicines containing LABA, as an overdose may result. Clinically significant cardiovascular effects and fatalities have been reported in association with excessive use of inhaled sympathomimetic drugs. Patients using AirDuo Digihaler should not use another medicine containing a LABA (e.g., salmeterol, formoterol fumarate, arformoterol tartrate, indacaterol) for any reason

Please see additional Safety Information throughout this brochure and full Prescribing Information for [ProAir Digihaler](#), [ArmonAir Digihaler](#) and [AirDuo Digihaler](#).

The Digihaler® Events Chart (2 of 3)

Here is an example selection of a 30-day view:

On the top left side of the report you are able to toggle between a 30- and 90-day view of patient data

Use the arrows to scroll through date ranges, which appear in increments of 30 and 90 days



IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Most common adverse reactions** ($\geq 1\%$ and $>$ placebo) are back pain, pain, gastroenteritis viral, sinus headache, urinary tract infection, nasopharyngitis, oropharyngeal pain and vomiting

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Oropharyngeal Candidiasis** has occurred in patients treated with ArmonAir Digihaler or AirDuo Digihaler. Advise patients to rinse the mouth with water without swallowing following inhalation

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

ACCOUNT
BASICS

INVITING
PATIENTS

NAVIGATING THE
PATIENT LIST

VIEWING PATIENT
DETAILS

EXPORTING
PATIENT DATA

SUPPORT AND
TROUBLESHOOTING

INVITING
TEAMMATES

PROGRAM
SETTINGS




The Digihaler® Events Chart (3 of 3)

Here is an example selection of a 90-day view:

Please Note that you can hover over the event column for more report details such as:

- The day or date range
- The total number of events
- The count of events by each inhalation category

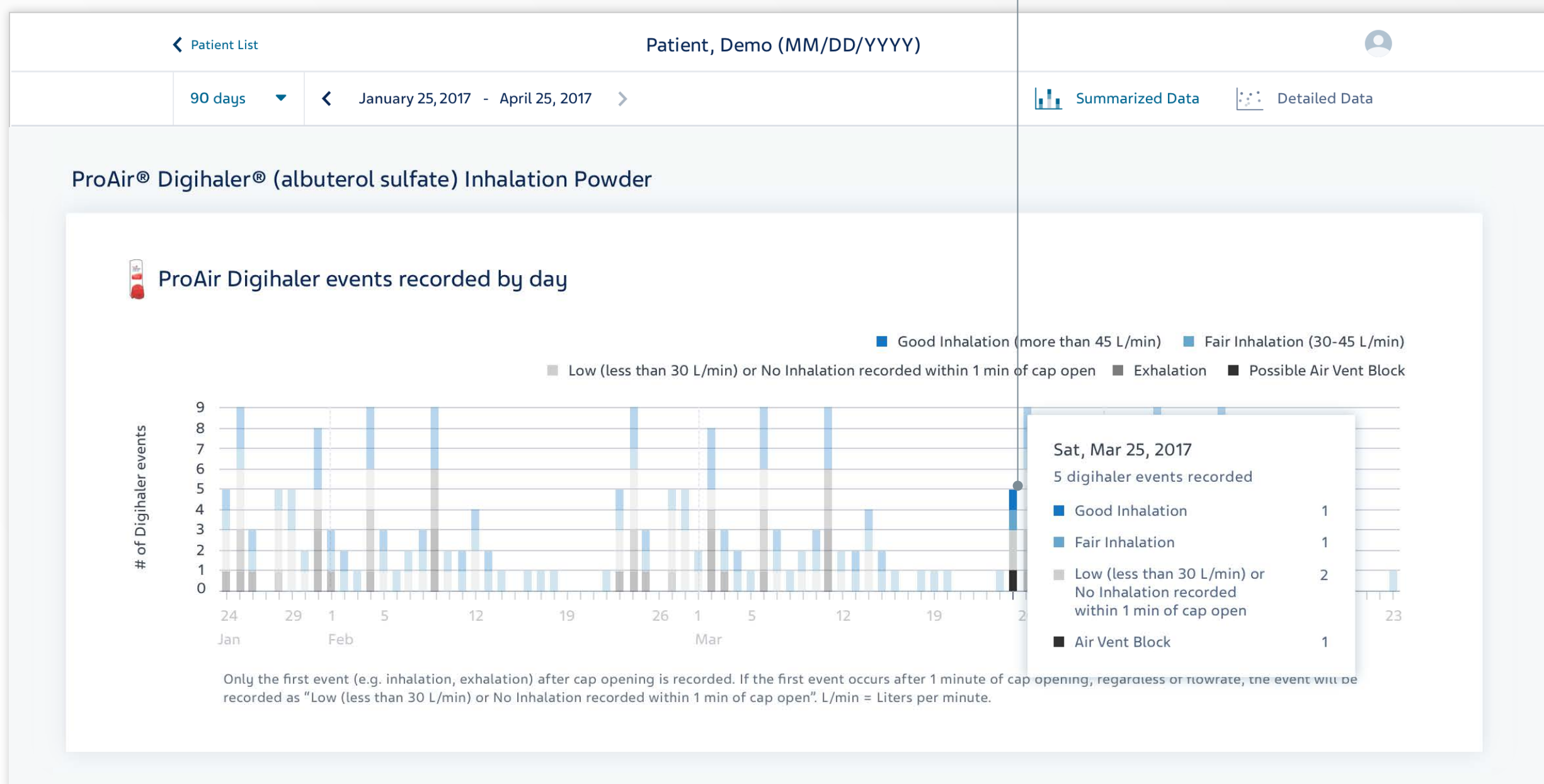

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** Other short-acting sympathomimetic bronchodilators should not be used concomitantly with ProAir Digihaler

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Immunosuppression and Risks of Infections:** Patients who use corticosteroids, such as found in AirDuo Digihaler and ArmonAir Digihaler are at risk for potential worsening of existing tuberculosis; fungal, bacterial, viral, or parasitic infections; or ocular herpes simplex. A more serious or even fatal course of chickenpox or measles may occur in susceptible patients. Use with caution in patients with the above because of the potential for worsening of these infections

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

The Recorded Digihaler® Events Table (1 of 9)

The Recorded Digihaler Events table is the second section of the Patient Details Screen, and provides a different view of patient data in list form. 30-day and 90-day views are available.

| Recorded Digihaler Events | | | What is a Digihaler Event? |
|---------------------------|--------------------|--------------------|----------------------------|
| Date | ProAir Digihaler ⓘ | | Daily Self-Assessment ⓘ |
| | Recorded Events | % Good Inhalations | |
| 04/17 - 04/23/2017 | 4 | 0% | 5 ▲ |
| Sat 04/23/2017 | 1 | 0% | ☹️ ▼ |
| Fri 04/22/2017 | - | - | - ▼ |
| Thurs 04/21/2017 | - | - | ☹️ ▼ |
| Wed 04/20/2017 | - | - | - ▼ |
| Tues 04/19/2017 | 1 | 0% | ☹️ ▼ |
| Mon 04/18/2017 | 1 | 0% | ☹️ ▼ |
| Sun 04/17/2017 | 1 | 0% | 😊 ▼ |
| 04/10 - 04/16/2017 | 20 | 30% | 3 ▼ |
| 04/03 - 04/09/2017 | 21 | 24% | 0 ▼ |
| 03/27 - 04/02/2017 | 26 | 19% | 2 ▼ |
| 03/25 - 03/26/2017 | 14 | 21% | 0 ▼ |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** (continued)
 - » **Beta-Blockers:** Beta-adrenergic-receptor blocking agents not only block the pulmonary effect of beta-agonists, such as ProAir Digihaler, but may produce severe bronchospasm in asthmatic patients. Therefore, patients with asthma should not normally be treated with beta-blockers

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Transferring Patients from Systemic Corticosteroid Therapy:** Particular care is needed for patients who have been transferred from systemically active corticosteroids to ICS because deaths due to adrenal insufficiency have occurred in patients with asthma during and after transfer from systemic corticosteroids to less systemically available ICS. Taper patients slowly from systemic corticosteroids if transferring to ArmonAir Digihaler or AirDuo Digihaler

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

The Recorded Digihaler® Events Table (2 of 9)

Here is an example selection of a collapsed 30-day view:

← Patient List
Patient, Demo (MM/DD/YYYY)
👤

30 days ▾
← March 25 2017 - April 23, 2017 →

📊 Summarized Data
🔍 Detailed Data

recorded as "Low (less than 30 L/min) or No Inhalation recorded within 1 min of cap open". L/min = Liters per minute.

Recorded Digihaler Events
[What is a Digihaler Event?](#)

| Date | 📌 ProAir Digihaler ? | | 📌 AirDuo Digihaler ? | | Daily Self-Assessment ? |
|--------------------|----------------------|--------------------|----------------------|--------------------|-------------------------|
| | Recorded Events | % Good Inhalations | Recorded Events | % Good Inhalations | |
| 04/17 - 04/23/2017 | 4 | 0% | - | - | 5 ▾ |
| 04/10 - 04/16/2017 | 20 | 30% | - | - | 3 ▾ |
| 04/03 - 04/09/2017 | 21 | 24% | 10 | 40% | 0 ▾ |
| 03/27 - 04/02/2017 | 26 | 19% | 9 | 55% | 2 ▾ |
| 03/25 - 03/26/2017 | 14 | 21% | 3 | 66% | 0 ▾ |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** (continued)
 - » **Diuretics:** Caution is advised in the coadministration of beta-agonists with non-potassium sparing diuretics (such as loop or thiazide diuretics). Consider monitoring potassium levels

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Hypercorticism and Adrenal Suppression** may occur with high doses of ICS, including fluticasone propionate, or at the recommended dose in susceptible individuals. If such changes occur, discontinue ArmonAir Digihaler or AirDuo Digihaler slowly

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

The Recorded Digihaler® Events Table (3 of 9)

To see an expanded 30-day view, click on any 7-day date range within this table to see the data for each individual day within the selected period.

| Patient List | | Patient, Demo (MM/DD/YYYY) | | | | This shows the number of Daily-Self-Assessments for the selected 7-day period | |
|---------------------------|-------------------------|--------------------------------|-------------------------|-----------------|---------------------------|---|----------------------------|
| 30 days | | March 25 2017 - April 23, 2017 | | Summarized Data | | Detailed Data | |
| Recorded Digihaler Events | | | | | | | What is a Digihaler Event? |
| Date | ProAir Digihaler Events | | AirDuo Digihaler Events | | ArmonAir Digihaler Events | | Daily Self-Assessment |
| | Events | % Good | Events | % Good | Events | % Good | |
| 05/14/2021 - 05/20/2021 | - | - | - | - | - | - | 0 |
| 05/07/2021 - 05/13/2021 | 4 | 100% | - | - | 7 | 100% | 4 |
| Thu 05/13/2021 | 3 | 100% | - | - | - | - | - |
| Wed 05/12/2021 | 1 | 100% | - | - | 1 | 100% | ☹️ |
| Tue 05/11/2021 | - | - | - | - | 2 | 100% | 😊 |
| Mon 05/10/2021 | - | - | - | - | 2 | 100% | 😐 |
| Sun 05/09/2021 | - | - | - | - | - | - | - |
| Sat 05/08/2021 | - | - | - | - | - | - | - |
| Fri 05/07/2021 | - | - | - | - | 2 | 100% | 😊 |
| 04/30/2021 - 05/06/2021 | 1 | 100% | - | - | 8 | 100% | 7 |
| 04/23/2021 - 04/29/2021 | 5 | 100% | - | - | 9 | 100% | 6 |
| 04/21/2021 - 04/22/2021 | - | - | - | - | 3 | 100% | 2 |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** (continued)
 - » **Digoxin:** Carefully evaluate the serum digoxin levels in patients who are currently receiving digoxin and ProAir Digihaler

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Drug Interactions with Strong Cytochrome P450 3A4 Inhibitors:** The use of strong cytochrome P450 3A4 (CYP3A4) inhibitors (e.g., ritonavir, ketoconazole) with ArmonAir Digihaler or AirDuo Digihaler is not recommended because increased systemic corticosteroid adverse effects may occur; increased cardiovascular adverse effects may also occur with AirDuo Digihaler

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

The Recorded Digihaler® Events Table (4 of 9)

Here is an example selection of a 90-day view:

Recorded Digihaler Events [What is a Digihaler Event?](#)

| Date | ProAir Digihaler ? | | AirDuo Digihaler ? | | Daily Self-Assessment ? |
|--------------------|--------------------|--------------------|--------------------|--------------------|-------------------------|
| | Recorded Events | % Good Inhalations | Recorded Events | % Good Inhalations | |
| 04/17 - 04/23/2017 | 4 | 0% | - | - | 5 |
| 04/10 - 04/16/2017 | 20 | 30% | - | - | 3 |
| 04/03 - 04/09/2017 | 21 | 24% | 10 | 40% | 0 |
| 03/27 - 04/02/2017 | 26 | 19% | 9 | 55% | 2 |
| 03/20 - 03/26/2017 | 34 | 26% | 2 | 0% | 0 |
| 03/13 - 03/19/2017 | 38 | 37% | 15 | 0% | 2 |
| 03/06 - 03/12/2017 | 7 | 57% | 8 | 50% | 0 |

[Show more](#)

Select "Show more" to see more date ranges

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** (continued)
 - » **Monoamine Oxidase Inhibitors or Tricyclic Antidepressants:** ProAir Digihaler should be administered with extreme caution to patients being treated with these agents, or within 2 weeks of discontinuation of these agents, because the action of albuterol on the cardiovascular system may be potentiated. Consider alternative therapy

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Paradoxical Bronchospasm and Upper Airway Symptoms:** Paradoxical bronchospasm may occur. If bronchospasm occurs treat immediately with an inhaled, short-acting bronchodilator discontinue AirDuo Digihaler or ArmonAir Digihaler and institute alternative therapy

Please see additional Safety Information throughout this brochure and full Prescribing Information for [ProAir Digihaler](#) [ArmonAir Digihaler](#) and [AirDuo Digihaler](#).

The Recorded Digihaler® Events Table (5 of 9)

Recorded events for each inhaler, percent of good inhalations, and average daily self assessments.

The table includes:

← Patient List
Patient, Demo (MM/DD/YYYY)
👤

90 days ▾
← January 25, 2017 - April 25, 2017 →
📊 Summarized Data
📄 Detailed Data

Recorded Digihaler Events What is a Digihaler Event?

| Date | 📦 ProAir Digihaler ? | | 📦 AirDuo Digihaler ? | | Daily Self-Assessment ? |
|--------------------|----------------------|--------------------|----------------------|--------------------|-------------------------|
| | Recorded Events | % Good Inhalations | Recorded Events | % Good Inhalations | |
| 04/17 - 04/23/2017 | 4 | 0% | - | - | 5 ▲ |
| Sat 04/23/2017 | 1 | 0% | - | - | 😐 ▼ |
| Fri 04/22/2017 | - | - | - | - | - ▼ |
| Thurs 04/21/2017 | - | - | - | - | 😞 ▼ |
| Wed 04/20/2017 | - | - | - | - | - ▼ |
| Tues 04/19/2017 | 1 | 0% | - | - | 😐 ▼ |
| Mon 04/18/2017 | 1 | 0% | - | - | 😞 ▼ |
| Sun 04/17/2017 | 1 | 0% | - | - | 😊 ▼ |
| 04/10 - 04/16/2017 | 20 | 30% | - | - | 3 ▼ |
| 04/03 - 04/09/2017 | 21 | 24% | 10 | 40% | 0 ▼ |
| 03/27 - 04/02/2017 | 26 | 19% | 9 | 55% | 2 ▼ |
| 03/25 - 03/26/2017 | 14 | 21% | 3 | 66% | 0 ▼ |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Paradoxical Bronchospasm:** ProAir Digihaler can produce paradoxical bronchospasm that may be life-threatening. Discontinue ProAir Digihaler and institute alternative therapy if paradoxical bronchospasm occurs

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Hypersensitivity Reactions, Including Anaphylaxis:** Immediate hypersensitivity reactions (e.g., urticaria, angioedema, rash, bronchospasm, hypotension), including anaphylaxis, may occur after administration of ArmonAir Digihaler or AirDuo Digihaler. Discontinue ArmonAir Digihaler or AirDuo Digihaler if such reactions occur

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

The Recorded Digihaler® Events Table (6 of 9)

Recorded events for each inhaler, percent of good inhalations, and average daily self assessments.

The table includes:

← Patient List
Patient, Demo (MM/DD/YYYY)
👤

90 days ▾
← January 25, 2017 - April 25, 2017 →
📊 Summarized Data
🔍 Detailed Data

Recorded Digihaler Events
What is a Digihaler Event?

| Date | 📱 ProAir Digihaler ? | | 📱 AirDuo Digihaler ? | | Daily Self-Assessment ? |
|--------------------|----------------------|--------------------|----------------------|--------------------|-------------------------|
| | Recorded Events | % Good Inhalations | Recorded Events | % Good Inhalations | |
| 04/17 - 04/23/2017 | 4 | 0% | - | - | 5 ▲ |
| Sat 04/23/2017 | 1 | 0% | - | - | ☹️ ▼ |
| Fri 04/22/2017 | - | - | - | - | - ▼ |
| Thurs 04/21/2017 | - | - | - | - | ☹️ ▼ |
| Wed 04/20/2017 | - | - | - | - | - ▼ |
| Tues 04/19/2017 | 1 | 0% | - | - | ☹️ ▼ |
| Mon 04/18/2017 | 1 | 0% | - | - | ☹️ ▼ |
| Sun 04/17/2017 | 1 | 0% | - | - | 😊 ▼ |
| 04/10 - 04/16/2017 | 20 | 30% | - | - | 3 ▼ |
| 04/03 - 04/09/2017 | 21 | 24% | 10 | 40% | 0 ▼ |
| 03/27 - 04/02/2017 | 26 | 19% | 9 | 55% | 2 ▼ |
| 03/25 - 03/26/2017 | 14 | 21% | 3 | 66% | 0 ▼ |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Deterioration of Asthma:** Need for more doses of ProAir Digihaler than usual may be a marker of acute or chronic deterioration of asthma and requires reevaluation of treatment, such as possible need for anti-inflammatory treatment, e.g., corticosteroids

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Cardiovascular and Central Nervous System Effects:** The salmeterol component of AirDuo Digihaler, can be associated with excessive beta-adrenergic stimulation which could present as the following symptoms: seizures, angina, hypertension or hypotension, tachycardia with rates up to 200 beats/min, arrhythmias, nervousness, headache, tremor, palpitation, nausea, dizziness, fatigue, malaise, and insomnia. Use with caution in patients with cardiac arrhythmias, hypertension, coronary insufficiency. Drug may need to be discontinued in certain patients.

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.


The Recorded Digihaler® Events Table (7 of 9)



Recorded events for each inhaler, percent of good inhalations, and average daily self assessments.

The table includes:






















% Good

The percentage of inhalations that were categorized as “Good Inhalations” (>45 L/min) out of the total number of Digihaler Events

← Patient List Patient, Demo (MM/DD/YYYY) 

90 days ▼ ← January 25, 2017 - April 25, 2017 →  Summarized Data  Detailed Data

Recorded Digihaler Events What is a Digihaler Event?

| Date | ProAir Digihaler  | | AirDuo Digihaler  | | Daily Self-Assessment  |
|--------------------|--|---|--|--------------------|---|
| | Recorded Events | % Good Inhalations | Recorded Events | % Good Inhalations | |
| 04/17 - 04/23/2017 | 4 | 0%  | - | - | 5  |
| Sat 04/23/2017 | 1 | 0% | - | - |   |
| Fri 04/22/2017 | - | - | - | - | -  |
| Thurs 04/21/2017 | - | - | - | - |   |
| Wed 04/20/2017 | - | - | - | - | -  |
| Tues 04/19/2017 | 1 | 0% | - | - |   |
| Mon 04/18/2017 | 1 | 0% | - | - |   |
| Sun 04/17/2017 | 1 | 0% | - | - |   |
| 04/10 - 04/16/2017 | 20 | 30% | - | - | 3  |
| 04/03 - 04/09/2017 | 21 | 24% | 10 | 40% | 0  |
| 03/27 - 04/02/2017 | 26 | 19% | 9 | 55% | 2  |
| 03/25 - 03/26/2017 | 14 | 21% | 3 | 66% | 0  |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Use of Anti-Inflammatory Agents:** ProAir Digihaler alone may not be adequate to control asthma in many patients. Early consideration should be given to adding anti-inflammatory agents, e.g., corticosteroids

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Reduction in Bone Mineral Density (BMD):** Decreases in BMD have been observed with long-term administration of products containing ICS. Patients with major risk factors for decreased bone mineral content, such as prolonged immobilization, family history of osteoporosis, or chronic use of drugs that can reduce bone mass (e.g., anticonvulsants, oral corticosteroids) should be monitored and treated with established standards of care when using ArmonAir Digihaler or AirDuo Digihaler

Please see additional Safety Information throughout this brochure and full Prescribing Information for [ProAir Digihaler](#) [ArmonAir Digihaler](#) and [AirDuo Digihaler](#).

The Recorded Digihaler® Events Table (8 of 9)

Recorded events for each inhaler, percent of good inhalations, and average daily self assessments.

The table includes:

Daily Self-Assessments

Patient records of how they felt each day, if the patient has chosen to complete assessments

| Patient List | | Patient, Demo (MM/DD/YYYY) | | | | Daily Self-Assessments | |
|---------------------------|------------------|-----------------------------------|------------------|--------------------|----------------------------|------------------------|--|
| 90 days | | January 25, 2017 - April 25, 2017 | | Summarized Data | | Detailed Data | |
| Recorded Digihaler Events | | | | | What is a Digihaler Event? | | |
| Date | ProAir Digihaler | | AirDuo Digihaler | | Daily Self-Assessment | | |
| | Recorded Events | % Good Inhalations | Recorded Events | % Good Inhalations | | | |
| 04/17 - 04/23/2017 | 4 | 0% | - | - | 5 | ▲ | |
| Sat 04/23/2017 | 1 | 0% | - | - | ☹️ | ▼ | |
| Fri 04/22/2017 | - | - | - | - | - | ▼ | |
| Thurs 04/21/2017 | - | - | - | - | ☹️ | ▼ | |
| Wed 04/20/2017 | - | - | - | - | - | ▼ | |
| Tues 04/19/2017 | 1 | 0% | - | - | ☹️ | ▼ | |
| Mon 04/18/2017 | 1 | 0% | - | - | ☹️ | ▼ | |
| Sun 04/17/2017 | 1 | 0% | - | - | 😊 | ▼ | |
| 04/10 - 04/16/2017 | 20 | 30% | - | - | 3 | ▼ | |
| 04/03 - 04/09/2017 | 21 | 24% | 10 | 40% | 0 | ▼ | |
| 03/27 - 04/02/2017 | 26 | 19% | 9 | 55% | 2 | ▼ | |
| 03/25 - 03/26/2017 | 14 | 21% | 3 | 66% | 0 | ▼ | |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Cardiovascular Effects:** ProAir Digihaler, like other beta-adrenergic agonists, can produce clinically significant cardiovascular effects in some patients, as measured by heart rate, blood pressure, and/or symptoms. If such effects occur, the drug may need to be discontinued. ProAir Digihaler, like all sympathomimetic amines, should be used with caution in patients with cardiovascular disorders, especially coronary insufficiency, cardiac arrhythmias, and hypertension

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Effect on Growth:** ICS may cause a reduction in growth velocity. Patients should be maintained on the lowest dose of inhaled corticosteroid that effectively controls their asthma. Monitor growth of pediatric patients receiving ArmonAir Digihaler and AirDuo Digihaler.

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

The Recorded Digihaler® Events Table (9 of 9)

Recorded events for each inhaler, percent of good inhalations, and average daily self assessments.

The table includes:

Daily Event Breakdown

From the dropdown arrow on the right side, expand the daily view to show a detailed view of each day.

| Patient List | | Patient, Demo (MM/DD/YYYY) | | | | |
|--|-----------------------------------|----------------------------|----|---------------|----|---|
| 90 days | January 25, 2017 - April 25, 2017 | Summarized Data | | Detailed Data | | |
| Fri 04/22/2017 | - | - | - | - | - | ▼ |
| Thurs 04/21/2017 | - | - | - | - | ☹️ | ▼ |
| Wed 04/20/2017 | - | - | - | - | - | ▼ |
| Tues 04/19/2017 | 1 | 0% | - | - | 😊 | ▼ |
| Mon 04/18/2017 | 1 | 0% | - | - | ☹️ | ▲ |
| Daily Event Breakdown for Mon 04/18/2017 | | | | | | |
| 1:42 PM ● Fair Inhalation (30-45 L/Min) ProAir® Digihaler® (albuterol sulfate), 117 mcg, Office | | | | | | |
| Sun 04/17/2017 | 1 | 0% | - | - | 😊 | ▼ |
| 04/10 - 04/16/2017 | 20 | 30% | - | - | 3 | ▼ |
| 04/03 - 04/09/2017 | 21 | 24% | 10 | 40% | 0 | ▼ |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Do Not Exceed Recommended Dose:** Fatalities have been reported in association with excessive use of inhaled sympathomimetic drugs in patients with asthma





IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Glaucoma and Cataracts:** Long-term use of ICS, including fluticasone propionate, a component of ArmonAir Digihaler and AirDuo Digihaler, may increase the risk for cataracts or glaucoma. Regular eye exams should be considered

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Digihaler® Connectivity (1 of 7)

The third section of the Patient Details Screen is where you can see a list of ProAir® Digihalers®, AirDuo® Digihalers®, and ArmonAir® Digihalers® that the patient has registered

| Digihaler Connectivity | | | | | Connectivity Help |
|--|----------------------------|------------|------------|------------------|-----------------------------------|
| Digihaler Type | Nickname & ID ? | Added ? | Removed ? | Last Connected ? | |
|  ProAir® Digihaler® (albuterol sulfate) Inhalation Powder, 117 mcg | Sports ID: XXXXXXXXXXXX | 03/21/2017 | — | 1 hour ago | |
|  ProAir® Digihaler® (albuterol sulfate) Inhalation Powder, 117 mcg | Travel ID: XXXXXXXXXXXX | 03/21/2017 | — | a month ago | |
|  AirDuo® Digihaler® (fluticasone propionate/salmeterol) Inhalation Powder, 113/14 mcg | April ID: XXXXXXXXXXXX | 04/01/2017 | — | 1 hour ago | |
|  AirDuo® Digihaler® (fluticasone propionate/salmeterol) Inhalation Powder, 113/14 mcg | May ID: XXXXXXXXXXXX | 03/01/2017 | 04/01/2017 | 22 days ago | |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Hypersensitivity Reactions including Anaphylaxis:** Immediate hypersensitivity reactions may occur after administration of albuterol sulfate, as demonstrated by rare cases of urticaria, angioedema, rash, bronchospasm, anaphylaxis, and oropharyngeal edema. Hypersensitivity reactions including anaphylaxis, angioedema, pruritus, and rash have been reported with the use of therapies containing lactose, an inactive ingredient in ProAir Digihaler.

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Eosinophilic Conditions and Churg-Strauss Syndrome:** Systemic eosinophilic conditions, such as Churg-Strauss syndrome, may occur when using ArmonAir Digihaler or AirDuo Digihaler. Be alert to eosinophilia, vasculitic rash, worsening pulmonary symptoms, cardiac complications, and/or neuropathy

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.





Digihaler® Connectivity (2 of 7)

The table includes:

Inhaler Type

The rescue or maintenance Digihaler that the patient has registered

Digihaler Connectivity Connectivity Help

| Digihaler Type | Nickname & ID ? | Added ? | Removed ? | Last Connected ? |
|--|----------------------------|------------|------------|------------------|
|  ProAir® Digihaler® (albuterol sulfate) Inhalation Powder, 117 mcg | Sports ID: XXXXXXXXXXXX | 03/21/2017 | — | 1 hour ago |
|  ProAir® Digihaler® (albuterol sulfate) Inhalation Powder, 117 mcg | Travel ID: XXXXXXXXXXXX | 03/21/2017 | — | a month ago |
|  AirDuo® Digihaler® (fluticasone propionate/salmeterol) Inhalation Powder, 113/14 mcg | April ID: XXXXXXXXXXXX | 04/01/2017 | — | 1 hour ago |
|  AirDuo® Digihaler® (fluticasone propionate/salmeterol) Inhalation Powder, 113/14 mcg | May ID: XXXXXXXXXXXX | 03/01/2017 | 04/01/2017 | 22 days ago |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Coexisting Conditions:** ProAir Digihaler, like all sympathomimetic amines, should be used with caution in patients with convulsive disorders, hyperthyroidism, or diabetes mellitus; and in patients who are unusually responsive to sympathomimetic amines.

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Coexisting Conditions:** Use AirDuo Digihaler with caution in patients with convulsive disorders, thyrotoxicosis, diabetes mellitus, ketoacidosis, and in patients who are unusually responsive to sympathomimetic amines

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.





Digihaler® Connectivity (3 of 7)

The table includes:

Nickname

The name the patient chose for the inhaler in the app

Digihaler Connectivity Connectivity Help

| Digihaler Type | Nickname & ID ? | Added ? | Removed ? | Last Connected ? |
|--|----------------------------|------------|------------|------------------|
|  ProAir® Digihaler® (albuterol sulfate) Inhalation Powder, 117 mcg | Sports ID: XXXXXXXXXXXX | 03/21/2017 | — | 1 hour ago |
|  ProAir® Digihaler® (albuterol sulfate) Inhalation Powder, 117 mcg | Travel ID: XXXXXXXXXXXX | 03/21/2017 | — | a month ago |
|  AirDuo® Digihaler® (fluticasone propionate/salmeterol) Inhalation Powder, 113/14 mcg | April ID: XXXXXXXXXXXX | 04/01/2017 | — | 1 hour ago |
|  AirDuo® Digihaler® (fluticasone propionate/salmeterol) Inhalation Powder, 113/14 mcg | May ID: XXXXXXXXXXXX | 03/01/2017 | 04/01/2017 | 22 days ago |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Hypokalemia:** As with other beta-agonists, ProAir Digihaler may produce significant hypokalemia in some patients. The decrease is usually transient, not requiring supplementation

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Hypokalemia and Hyperglycemia:** Beta-adrenergic agonist medicines may produce significant hypokalemia in some patients, possibly through intracellular shunting, which has the potential to produce adverse cardiovascular effects. Decrease in serum potassium are usually transient, not requiring supplementation. Be alert to hypokalemia and hyperglycemia in patients using AirDuo Digihaler

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Digihaler® Connectivity (4 of 7)

The table includes:

ID

The ID number of the inhaler found on the QR code



| Digihaler Connectivity | | | | | Connectivity Help |
|--|----------------------------|------------|------------|------------------|-----------------------------------|
| Digihaler Type | Nickname & ID ? | Added ? | Removed ? | Last Connected ? | |
| ProAir® Digihaler® (albuterol sulfate) Inhalation Powder, 117 mcg | Sports ID: XXXXXXXXXXXX | 03/21/2017 | — | 1 hour ago | |
| ProAir® Digihaler® (albuterol sulfate) Inhalation Powder, 117 mcg | Travel ID: XXXXXXXXXXXX | 03/21/2017 | — | a month ago | |
| AirDuo® Digihaler® (fluticasone propionate/salmeterol) Inhalation Powder, 113/14 mcg | April ID: XXXXXXXXXXXX | 04/01/2017 | — | 1 hour ago | |
| AirDuo® Digihaler® (fluticasone propionate/salmeterol) Inhalation Powder, 113/14 mcg | May ID: XXXXXXXXXXXX | 03/01/2017 | 04/01/2017 | 22 days ago | |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Most common adverse reactions** ($\geq 1\%$ and $>$ placebo) are back pain, pain, gastroenteritis viral, sinus headache, urinary tract infection, nasopharyngitis, oropharyngeal pain and vomiting

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Adverse Reactions with ArmonAir Digihaler:** Most common adverse reactions (greater than or equal to 3%) are: upper respiratory tract infection, nasopharyngitis, oral candidiasis, headache, and cough





Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Digihaler® Connectivity (5 of 7)

The table includes:

Added

The date the inhaler was first registered

| Digihaler Connectivity | | | | | Connectivity Help |
|--|----------------------------|------------|------------|------------------|-------------------|
| Digihaler Type | Nickname & ID ? | Added ? | Removed ? | Last Connected ? | |
|  ProAir® Digihaler® (albuterol sulfate) Inhalation Powder, 117 mcg | Sports ID: XXXXXXXXXXXX | 03/21/2017 | — | 1 hour ago | |
|  ProAir® Digihaler® (albuterol sulfate) Inhalation Powder, 117 mcg | Travel ID: XXXXXXXXXXXX | 03/21/2017 | — | a month ago | |
|  AirDuo® Digihaler® (fluticasone propionate/salmeterol) Inhalation Powder, 113/14 mcg | April ID: XXXXXXXXXXXX | 04/01/2017 | — | 1 hour ago | |
|  AirDuo® Digihaler® (fluticasone propionate/salmeterol) Inhalation Powder, 113/14 mcg | May ID: XXXXXXXXXXXX | 03/01/2017 | 04/01/2017 | 22 days ago | |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** Other short-acting sympathomimetic bronchodilators should not be used concomitantly with ProAir Digihaler

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Adverse Reactions with AirDuo Digihaler:** Most common adverse reactions (greater than or equal to 3%) include nasopharyngitis, oral candidiasis, headache, cough and back pain

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.





Digihaler® Connectivity (6 of 7)

The table includes:

Removed

The date the inhaler was removed from the app

Please Note that a dash indicates the inhaler is currently registered

| Digihaler Connectivity | | | | | Connectivity Help |
|--|----------------------------|------------|------------|------------------|-------------------|
| Digihaler Type | Nickname & ID ? | Added ? | Removed ? | Last Connected ? | |
|  ProAir® Digihaler® (albuterol sulfate) Inhalation Powder, 117 mcg | Sports ID: XXXXXXXXXXXX | 03/21/2017 | — | 1 hour ago | |
|  ProAir® Digihaler® (albuterol sulfate) Inhalation Powder, 117 mcg | Travel ID: XXXXXXXXXXXX | 03/21/2017 | — | a month ago | |
|  AirDuo® Digihaler® (fluticasone propionate/salmeterol) Inhalation Powder, 113/14 mcg | April ID: XXXXXXXXXXXX | 04/01/2017 | — | 1 hour ago | |
|  AirDuo® Digihaler® (fluticasone propionate/salmeterol) Inhalation Powder, 113/14 mcg | May ID: XXXXXXXXXXXX | 03/01/2017 | 04/01/2017 | 22 days ago | |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** (continued)
 - » **Beta-Blockers:** Beta-adrenergic-receptor blocking agents not only block the pulmonary effect of beta-agonists, such as ProAir Digihaler, but may produce severe bronchospasm in asthmatic patients. Therefore, patients with asthma should not normally be treated with beta-blockers





IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Serious Asthma-Related Events – Hospitalizations, Intubations, Death:** Use of a LABA as monotherapy (without an ICS) for asthma is associated with an increased risk of asthma-related death. Available data from controlled clinical trials also suggest that use of LABA as monotherapy increases the risk of asthma-related hospitalization in pediatric and adolescent patients. These findings are considered a class effect of LABA monotherapy. When LABA are used in fixed-dose combination with ICS (such as AirDuo Digihaler), data from large clinical trials do not show a significant increase in the risk of serious asthma-related events (hospitalizations, intubations, death) compared with ICS alone

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Digihaler® Connectivity (7 of 7)

The table includes:

| Digihaler Connectivity | | | | | Connectivity Help |
|--|----------------------------|------------|------------|------------------|-------------------|
| Digihaler Type | Nickname & ID ? | Added ? | Removed ? | Last Connected ? | |
|  ProAir® Digihaler® (albuterol sulfate) Inhalation Powder, 117 mcg | Sports ID: XXXXXXXXXXXX | 03/21/2017 | — | 1 hour ago | |
|  ProAir® Digihaler® (albuterol sulfate) Inhalation Powder, 117 mcg | Travel ID: XXXXXXXXXXXX | 03/21/2017 | — | a month ago | |
|  AirDuo® Digihaler® (fluticasone propionate/salmeterol) Inhalation Powder, 113/14 mcg | April ID: XXXXXXXXXXXX | 04/01/2017 | — | 1 hour ago | |
|  AirDuo® Digihaler® (fluticasone propionate/salmeterol) Inhalation Powder, 113/14 mcg | May ID: XXXXXXXXXXXX | 03/01/2017 | 04/01/2017 | 22 days ago | |

Last Connected

Indicates the last known time that the patient's ProAir® Digihalers®, AirDuo® Digihalers®, and/or ArmonAir® Digihalers® were connected to the app

Please Note that this information will only be the most up-to-date if the inhaler is connected to the Digihaler App and the patient's Bluetooth® Wireless Technology is turned on and within range

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** (continued)
 - » **Diuretics:** Caution is advised in the coadministration of beta-agonists with non-potassium sparing diuretics (such as loop or thiazide diuretics). Consider monitoring potassium levels


IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Deterioration of Disease and Acute Episodes:** AirDuo Digihaler should not be initiated in patients during rapidly deteriorating or potentially life-threatening episodes of asthma. ArmonAir Digihaler and AirDuo Digihaler are not indicated for the relief of acute bronchospasm. An inhaled, short-acting beta₂-agonist, not ArmonAir Digihaler or AirDuo Digihaler, should be used to relieve acute symptoms such as shortness of breath

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.


Accessing Detailed Data

At the top right of the Patient Details Screen, you will be able to toggle between the "Summarized Data" and "Detailed Data" screens.

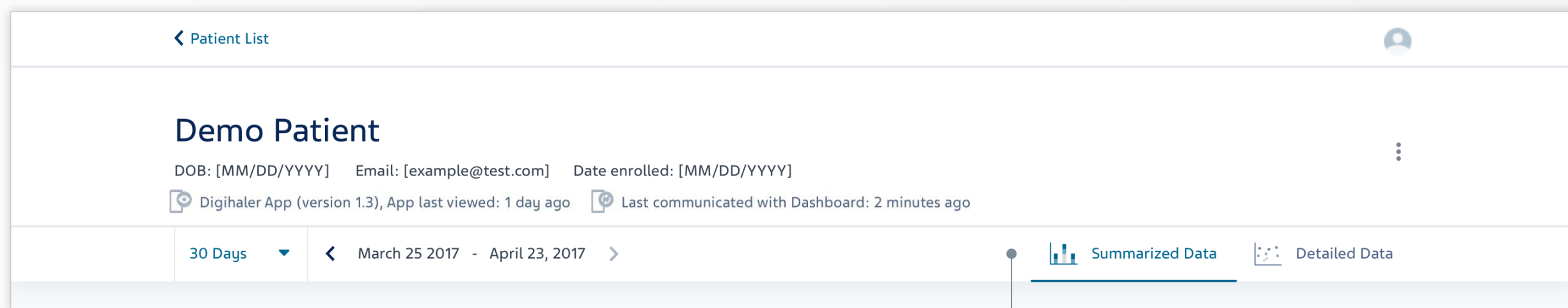

By default, clicking on a patient's name from the Patient List will take you to the "Summarized Data" screen within the Patient Details page. The "Summarized Data" screen displays the Digihaler® Events chart (shown on page 36).

By clicking on the "Detailed Data" button, you will be able to toggle between the Digihaler Events chart and charts that display Peak Inspiratory Flow (PIF) and Inhalation Volume by medication type (shown on pages 56 and 57).

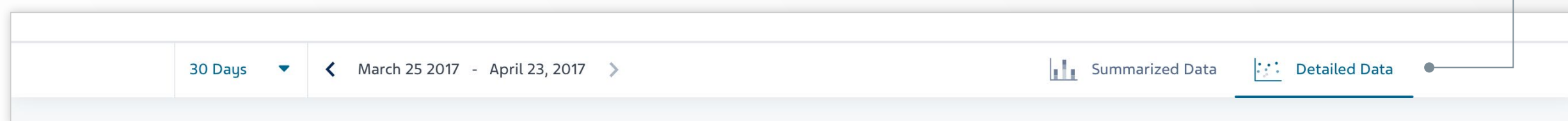

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** (continued)
 - » **Digoxin:** Carefully evaluate the serum digoxin levels in patients who are currently receiving digoxin and ProAir Digihaler


IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Avoid Excessive Use and Avoid Use with Other Long acting Beta2-Agonists:** AirDuo Digihaler should not be used more often than recommended, at higher doses than recommended, or in conjunction with other medicines containing LABA, as an overdose may result. Clinically significant cardiovascular effects and fatalities have been reported in association with excessive use of inhaled sympathomimetic drugs.
- Patients using AirDuo Digihaler should not use another medicine containing a LABA (e.g., salmeterol, formoterol fumarate, arformoterol tartrate, indacaterol) for any reason

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

The Peak Inspiratory Flow Chart

The **Peak Inspiratory Flow** (PIF) chart will present specific Peak Inspiratory Flow Rate values in Liters/minute over a 30 or 90-day period from 30-90 L/min (ProAir® Digihaler®) and 30-120 L/min (AirDuo® Digihaler® and ArmonAir® Digihaler®). Inhalations that fall outside this range are counted in distinct categories surrounding the chart.

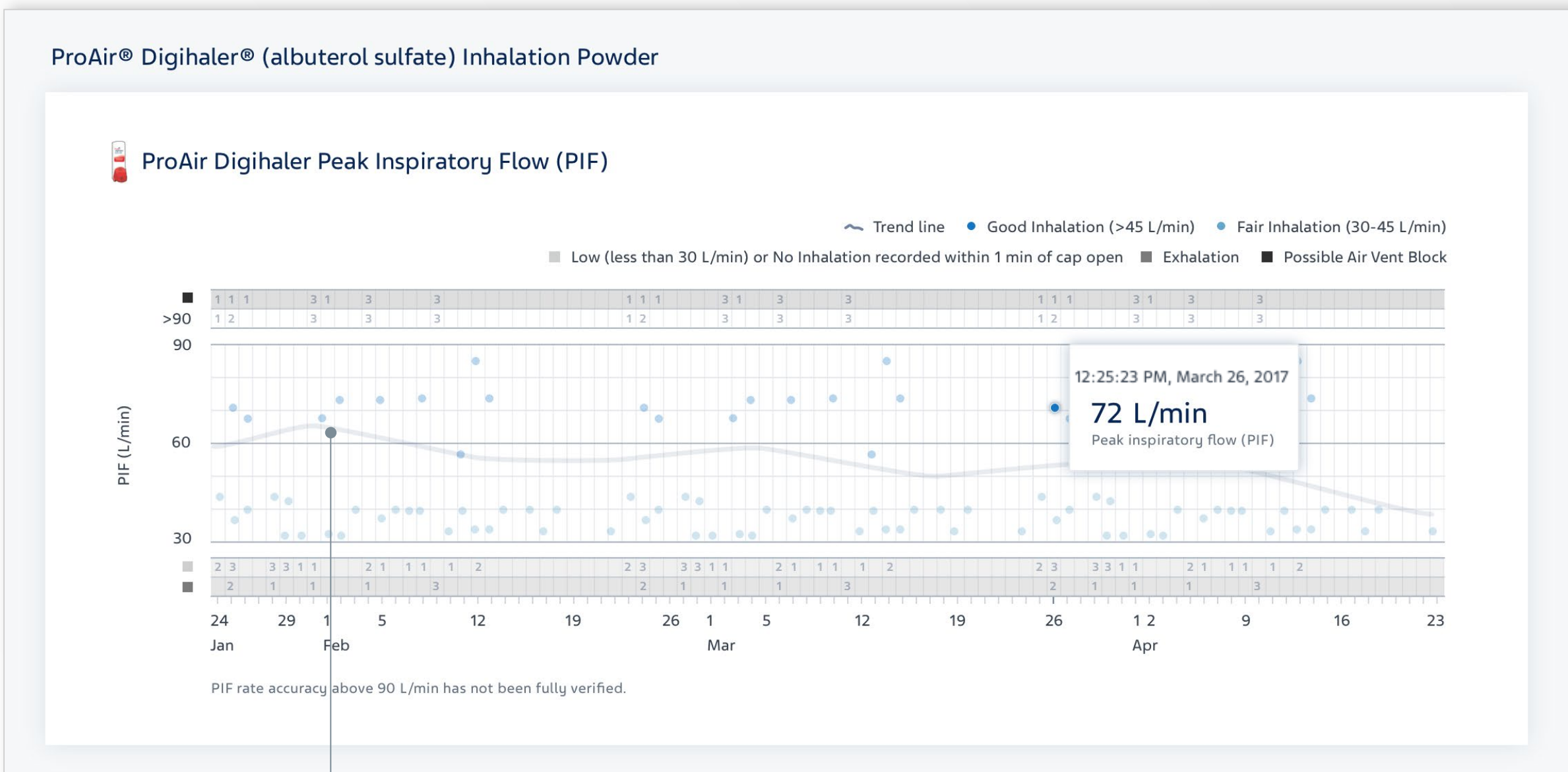

Provided at least 10 inhalations are plotted, a trend line will be displayed to help the user identify trends in the values. The trend line will use a locally estimated scatterplot smoothing algorithm (LOESS) with a smoothing factor of 0.5. PIF rate accuracy above 90 L/min has not been fully verified.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** (continued)
 - » **Monoamine Oxidase Inhibitors or Tricyclic Antidepressants:** ProAir Digihaler should be administered with extreme caution to patients being treated with these agents, or within 2 weeks of discontinuation of these agents, because the action of albuterol on the cardiovascular system may be potentiated. Consider alternative therapy

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Oropharyngeal Candidiasis** has occurred in patients treated with ArmonAir Digihaler or AirDuo Digihaler. Advise patients to rinse the mouth with water without swallowing following inhalation

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

The Inhalation Volume Chart

The **Inhalation Volume** chart will present specific Inhalation Volume values in Liters over a 30 or 90-day period, based on Digihaler® events from 0.4 - 4.0 L. Inhalations that fall outside this range are counted in distinct categories surrounding the chart.



Provided at least 10 inhalations are plotted, a trend line will be displayed to help the user identify trends in the values. The trend line will use a locally estimated scatterplot smoothing algorithm (LOESS) with a smoothing factor of 0.5.

Upon hovering over each data point, you will see a reading for the inhalation volume on a given date and time. There are two sets of numbers, the calculated volume as well as an estimated range.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Paradoxical Bronchospasm:** ProAir Digihaler can produce paradoxical bronchospasm that may be life-threatening. Discontinue ProAir Digihaler and institute alternative therapy if paradoxical bronchospasm occurs

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Immunosuppression and Risks of Infection:** Patients who use corticosteroids, such as found in AirDuo Digihaler and ArmonAir Digihaler are at risk for potential worsening of existing tuberculosis; fungal, bacterial, viral, or parasitic infections; or ocular herpes simplex. A more serious or even fatal course of chickenpox or measles may occur in susceptible patients. Use with caution in patients with the above because of the potential for worsening of these infections

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Section 5: Exporting Patient Data

Setting up an Export (1 of 4)

From the Patient List, the Digihaler® Dashboard allows you to export Patient Inhaler Event Data for all of your patients enrolled in the program. This allows healthcare providers to download reports and bring them to consultations to view with their patients.

Please Note that the file contains patient data and the user should handle this information per their policies

To export a report, click “Export Data” at the top right side of the “Patient List”

The screenshot shows the Digihaler Patient List interface. At the top, there are logos for ProAir Digihaler (albuterol sulfate inhalation powder), AirDuo Digihaler (fluticasone propionate and salmeterol inhalation powder), and ArmonAir Digihaler (fluticasone propionate inhalation powder). Below the logos, there are tabs for 'PRESCRIBING INFORMATION' and 'IMPORTANT SAFETY INFORMATION'. The main heading is 'Patient List' with an 'Invite Patient' button on the right. Below the heading, there is a search bar 'Search list by name' and a 'Filter' button. A table lists 10 patients with columns for 'Last, First', 'DOB', and three inhaler categories: 'ProAir Digihaler (Last 7 Days)', 'AirDuo Digihaler (Last 30 Days)', and 'ArmonAir Digihaler (Last 30 Days)'. Each category has sub-columns for 'Events', '% Good', and 'All Connected'. An 'Export Data' button is located at the top right of the table area.

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|-----------------|------------|--------------------------------|--------|----------------|---------------------------------|--------|----------------|-----------------------------------|--------|---------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1958 | 40 | 38% | 30 minutes ago | 22 | 61% | 1 hour ago | - | - | - |
| Demo, Patient 2 | 04/26/1964 | 17 | 100% | 5 hours ago | - | - | - | 44 | 100% | 5 hours ago |
| Demo, Patient 3 | 03/22/1966 | 12 | 83% | 23 hours ago | 33 | 97% | 3 hours ago | - | - | - |
| Demo, Patient 4 | 11/14/1978 | 5 | 60% | 20 minutes ago | 43 | 93% | 20 minutes ago | - | - | - |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Deterioration of Asthma:** Need for more doses of ProAir Digihaler than usual may be a marker of acute or chronic deterioration of asthma and requires reevaluation of treatment, such as possible need for anti-inflammatory treatment, e.g., corticosteroids

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Transferring Patients from Systemic Corticosteroid Therapy:** Particular care is needed for patients who have been transferred from systemically active corticosteroids to ICS because deaths due to adrenal insufficiency have occurred in patients with asthma during and after transfer from systemic corticosteroids to less systemically available ICS. Taper patients slowly from systemic corticosteroids if transferring to ArmonAir Digihaler or AirDuo Digihaler

Please see additional Safety Information throughout this brochure and full Prescribing Information for [ProAir Digihaler](#) [ArmonAir Digihaler](#) and [AirDuo Digihaler](#).

Setting up an Export (2 of 4)

You will be prompted to select the date range for all patients' inhaler event data.

No selection is made by default.

You can click "All Time" which will result in all data being downloaded from the start of the program (as specified when your specific program was created) to the current date.

Export all patient data

Select the date range you would like to export:

All time

Custom

Please note that exporting data for all patients may take some time. You can continue to use the dashboard while the export is in progress. Once the export is complete, the download will only be available for 15 minutes.

[Export CSV](#)

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Use of Anti-Inflammatory Agents:** ProAir Digihaler alone may not be adequate to control asthma in many patients. Early consideration should be given to adding anti-inflammatory agents, e.g., corticosteroids

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Hypercorticism and Adrenal Suppression** may occur with high doses of ICS, including fluticasone propionate, or at the recommended dose in susceptible individuals. If such changes occur, discontinue ArmonAir Digihaler or AirDuo Digihaler slowly

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Setting up an Export (3 of 4)

You will be prompted to select the date range for all patients' inhaler event data. No selection is made by default.

Export all patient data

Select the date range you would like to export:

All time

Custom

04/23/2017 04/23/2017

Please note that exporting data for use the dashboard while the export download will only be available for

April 2017

| SU | MO | TU | WE | TH | FR | SA |
|----|----|----|----|----|----|----|
| 26 | 27 | 28 | 29 | 30 | 31 | 1 |
| 2 | 3 | 4 | 5 | 6 | 7 | 8 |
| 9 | 10 | 11 | 12 | 13 | 14 | 15 |

Once the date range has been selected, you should click the "Export CSV" button

Export CSV

You can also click a "Custom" date range

Please Note that the start date time is considered 12:00AM while the end date time is considered 11:59PM

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Cardiovascular Effects:** ProAir Digihaler, like other beta-adrenergic agonists, can produce clinically significant cardiovascular effects in some patients, as measured by heart rate, blood pressure, and/or symptoms. If such effects occur, the drug may need to be discontinued. ProAir Digihaler, like all sympathomimetic amines, should be used with caution in patients with cardiovascular disorders, especially coronary insufficiency, cardiac arrhythmias, and hypertension

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Drug Interactions with Strong Cytochrome P450 3A4 Inhibitors:** The use of strong cytochrome P450 3A4 (CYP3A4) inhibitors (e.g., ritonavir, ketoconazole) with ArmonAir Digihaler or AirDuo Digihaler is not recommended because increased systemic corticosteroid adverse effects may occur; increased cardiovascular adverse effects may also occur with AirDuo Digihaler

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Setting up an Export (4 of 4)

You will then see “Exporting Data” at the top right side of your Dashboard view.

“Exporting Data” in progress

Patient List

Invite Patient

10 Patients Filter Exporting Data... 1 - 10 of 10

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|---|------------|--------------------------------|--------|----------------|---------------------------------|--------|---------------|-----------------------------------|--------|---------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1956 | 40 | 38% | 30 minutes ago | 22 | 61% | 1 hour ago | | | |
| Data export in progress. Cancel. | | | | | | | | | | |
| Demo, Patient 3 | 03/22/1966 | 12 | 83% | 23 hours ago | 33 | 97% | 3 hours ago | | | |

Once the exported Inhaler Event Report has been generated in the Dashboard, it is ready for download and made available via a download link. You will see the file appear in your “Downloads” folder

| | | | | | | | | | | |
|--|------------|---|---|------------|---|---|------------|---|---|--------------|
| Demo, Patient | 08/06/1956 | 0 | - | 3 days ago | 0 | - | 2 days ago | 0 | - | 2 months ago |
| Data export complete - the link to download will expire in [00:00]. Download. | | | | | | | | | | |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Do Not Exceed Recommended Dose:** Fatalities have been reported in association with excessive use of inhaled sympathomimetic drugs in patients with asthma

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Paradoxical Bronchospasm and Upper Airway Symptoms:** Paradoxical bronchospasm may occur. if bronchospasm occurs treat immediately with an inhaled, short-acting bronchodilator discontinue AirDuo Digihaler or ArmonAir Digihaler and institute alternative therapy

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Applying Filters

Rather than exporting data for all patients, you have the option to apply filters to the Patient List and export data for a smaller group. Search by patient name if you would like to export data for an individual patient.

Please Note: Once a filter is applied, the user can click an X icon to clear the filter and return to the Patient List containing all patients.

The number of patients in the list will adjust automatically based on the filter(s) applied.

By default, All Patients is selected, which displays the unfiltered list of patients

The screenshot shows the 'Patient List' interface. At the top, there are logos for ProAir Digihaler, AirDuo Digihaler, and ArmonAir Digihaler, along with links for 'PRESCRIBING INFORMATION' and 'IMPORTANT SAFETY INFORMATION'. A sidebar on the left contains 'Demo Program', 'Patient List', 'Team List', and 'Program Settings'. The main area displays '125 Patients' and a search bar. A table lists patient data with columns for 'Last, First', 'DOB', and three medication categories: 'ProAir Digihaler (Last 7 Days)', 'AirDuo Digihaler (Last 30 Days)', and 'ArmonAir Digihaler (Last 30 Days)'. Each medication category has sub-columns for 'Events', '% Good', and 'All Connected'. A 'Filter' button is highlighted with a callout.

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|-----------------|------------|--------------------------------|--------|----------------|---------------------------------|--------|----------------|-----------------------------------|--------|---------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1958 | 40 | 38% | 30 minutes ago | 22 | - | - | - | - | - |
| Demo, Patient 2 | 04/26/1964 | 17 | 100% | 5 hours ago | - | - | - | 44 | 100% | - |
| Demo, Patient 3 | 03/22/1966 | 12 | 83% | 23 hours ago | - | - | 2 months ago | - | - | - |
| Demo, Patient 4 | 11/14/1978 | 5 | 60% | 20 minutes ago | 43 | - | 20 minutes ago | - | - | - |

Clicking on Filter shows the user what filters are available.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Hypersensitivity Reactions including Anaphylaxis:** Immediate hypersensitivity reactions may occur after administration of albuterol sulfate, as demonstrated by rare cases of urticaria, angioedema, rash, bronchospasm, anaphylaxis, and oropharyngeal edema. Hypersensitivity reactions including anaphylaxis, angioedema, pruritus, and rash have been reported with the use of therapies containing lactose, an inactive ingredient in ProAir Digihaler.

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Hypersensitivity Reactions, Including Anaphylaxis:** Immediate hypersensitivity reactions (e.g., urticaria, angioedema, rash, bronchospasm, hypotension), including anaphylaxis, may occur after administration of ArmonAir Digihaler or AirDuo Digihaler. Discontinue ArmonAir Digihaler or AirDuo Digihaler if such reactions occur

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Section 6: Support and Troubleshooting

If you need further assistance with the Digihaler® Dashboard, or if you have any questions or concerns:

- Visit our [Support Center](#)
- Call customer assistance at 1-(833) 390-2147

If you're having issues accessing the Dashboard, check that your browser/system requirements meet the guidelines below:

Browser Requirements



Chrome
Version 70
or higher



Internet Explorer
Version 11



Edge
Version 17
or higher




Safari
Version 11
or higher



Firefox
Version 58
or higher

System Requirements

- 1110 pixel width is required to view all data.
- It is recommended that your screen resolution be set to 1280 x 800.
- U.S. English is the only supported language.
- The time is hard-coded to a 12-hour AM/PM display.

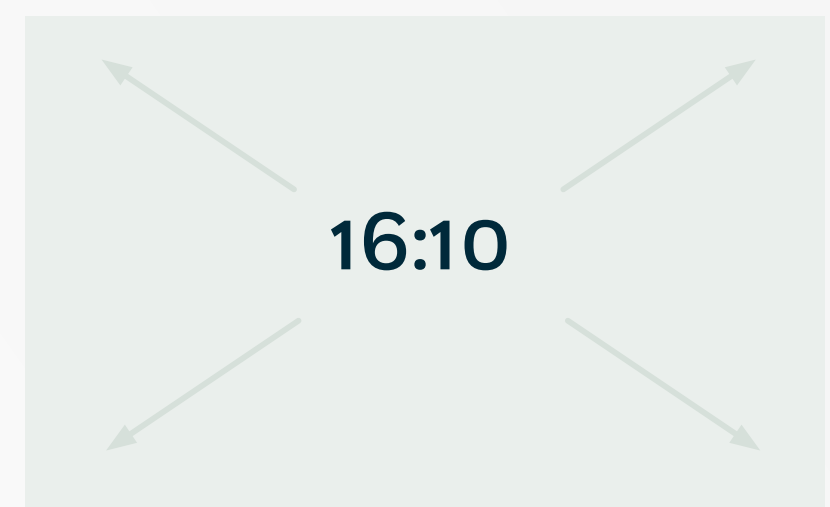

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Coexisting Conditions:** ProAir Digihaler, like all sympathomimetic amines, should be used with caution in patients with convulsive disorders, hyperthyroidism, or diabetes mellitus; and in patients who are unusually responsive to sympathomimetic amines.

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Cardiovascular and Central Nervous System Effects:** The salmeterol component of AirDuo Digihaler, can be associated with excessive beta-adrenergic stimulation which could present as the following symptoms: seizures, angina, hypertension or hypotension, tachycardia with rates up to 200 beats/min, arrhythmias, nervousness, headache, tremor, palpitation, nausea, dizziness, fatigue, malaise, and insomnia. Use with caution in patients with cardiac arrhythmias, hypertension, coronary insufficiency. Drug may need to be discontinued in certain patients.

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

ACCOUNT
BASICS

INVITING
PATIENTS

NAVIGATING THE
PATIENT LIST

VIEWING PATIENT
DETAILS

EXPORTING
PATIENT DATA

**SUPPORT AND
TROUBLESHOOTING**

INVITING
TEAMMATES

PROGRAM
SETTINGS

For Dashboard Admins and Super Admins

Section 7: Inviting Teammates

Managing your Program Together (1 of 4)

You can invite teammates to help you manage your specific program. Different teammates are provided different permissions ([See Dashboard Team section](#)). Be sure to choose teammates who will be the best fit for each role. When you log into your program, "Patient List" will be your default view (as shown).

Switch between "Patient List" and "Team List" from the left hand navigation menu

The screenshot shows a dashboard for a "Demo Program" with an email "example-email@test.com". The left navigation menu includes "Patient List" (selected), "Team List", "Program Settings", "About", "Help", "Privacy Notice", "Terms of Use", "Audit Log", and "Log Out". The main content area is titled "Patient List" and features an "Invite Patient" button. Below the title, it indicates "125 Patients" and includes a search bar, a "Filter" button, and an "Export Data" button. The patient list table has the following columns: Last, First; DOB; ProAir Digihaler (Last 7 Days) with sub-columns for Events, % Good, and All Connected; AirDuo Digihaler (Last 30 Days) with sub-columns for Events, % Good, and All Connected; and ArmonAir Digihaler (Last 30 Days) with sub-columns for Events, % Good, and All Connected. The table lists 11 demo patients with their respective medication usage statistics.

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|------------------|------------|--------------------------------|--------|----------------|---------------------------------|--------|----------------|-----------------------------------|--------|----------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1958 | 40 | 38% | 30 minutes ago | 22 | - | - | - | - | - |
| Demo, Patient 2 | 04/26/1964 | 17 | 100% | 5 hours ago | - | - | - | 44 | 100% | - |
| Demo, Patient 3 | 03/22/1966 | 12 | 83% | 23 hours ago | - | - | 2 months ago | - | - | - |
| Demo, Patient 4 | 11/14/1978 | 5 | 60% | 20 minutes ago | 43 | - | 20 minutes ago | - | - | - |
| Demo, Patient 5 | 02/06/1947 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 6 | 01/11/1984 | 4 | 0% | 30 minutes ago | - | - | - | 0 | - | 15 minutes ago |
| Demo, Patient 7 | 09/05/1957 | 3 | 0% | 3 days ago | - | - | - | - | 64% | 3 days ago |
| Demo, Patient 8 | 10/24/1999 | 0 | - | 3 hours ago | 10 | 30% | 3 hours ago | 10 | 0% | 2 days ago |
| Demo, Patient 9 | 04/16/1950 | 0 | - | 3 hours ago | 10 | 30% | 4 days ago | - | - | - |
| Demo, Patient 10 | 02/04/1948 | 0 | - | 1 month ago | 10 | 30% | 5 days ago | 0 | - | 1 month ago |
| Demo, Patient 11 | 09/21/1997 | 0 | - | 3 days ago | - | - | - | 0 | - | 2 months ago |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Hypokalemia:** As with other beta-agonists, ProAir Digihaler may produce significant hypokalemia in some patients. The decrease is usually transient, not requiring supplementation


IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Reduction in Bone Mineral Density (BMD):** Decreases in BMD have been observed with long-term administration of products containing ICS. Patients with major risk factors for decreased bone mineral content, such as prolonged immobilization, family history of osteoporosis, or chronic use of drugs that can reduce bone mass (e.g., anticonvulsants, oral corticosteroids) should be monitored and treated with established standards of care when using ArmonAir Digihaler or AirDuo Digihaler


Please see additional Safety Information throughout this brochure and full Prescribing Information for [ProAir Digihaler](#) [ArmonAir Digihaler](#) and [AirDuo Digihaler](#).


Managing your program together (2 of 4)


After the Super Administrator or Administrator on the team has **set up their profile and signed in, they can begin to invite other teammates:**

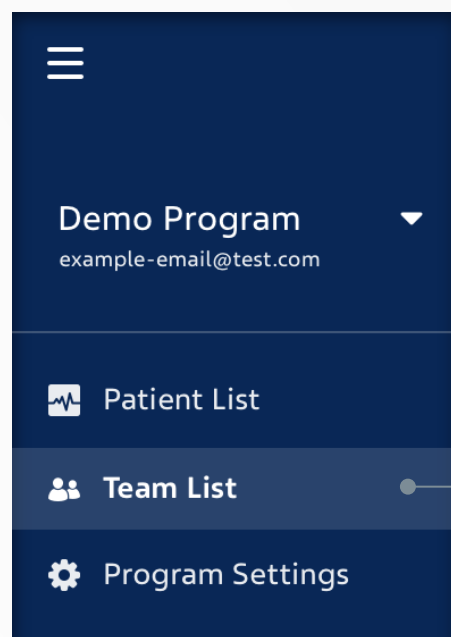

Make sure that you have “Team List” selected to invite teammates.


(albuterol sulfate)
inhalation powder


(fluticasone propionate and
salmeterol) inhalation powder


(fluticasone propionate)
inhalation powder

PRESCRIBING INFORMATION ▾ IMPORTANT SAFETY INFORMATION ▾

Team List

Invite Teammate

Teammates (6)

Pending Invitations (10)

6 Teammates

| Last, First ↕ | Email Address ↕ | Role ↕ | Date Enrolled ↕ |
|------------------|-------------------------|------------------|--|
| Demo, Teammate 1 | demo.teammate1@test.com | Super-Admin | 05/09/2017 |
| Demo, Teammate 2 | demo.teammate2@test.com | Admin | 06/23/2017 |
| Demo, Teammate 3 | demo.teammate3@test.com | Super-Admin | 05/20/2017 |
| Demo, Teammate 4 | demo.teammate4@test.com | Dashboard User ▾ | 06/01/2017 Remove |
| Demo, Teammate 5 | demo.teammate5@test.com | Dashboard User ▾ | 06/25/2017 Remove |
| Demo, Teammate 6 | demo.teammate6@test.com | Dashboard User ▾ | 06/30/2017 Remove |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Most common adverse reactions** ($\geq 1\%$ and $>$ placebo) are back pain, pain, gastroenteritis viral, sinus headache, urinary tract infection, nasopharyngitis, oropharyngeal pain and vomiting

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Effect on Growth:** ICS may cause a reduction in growth velocity, Patients should be maintained on the lowest dose of inhaled corticosteroid that effectively controls their asthma. Monitor growth of pediatric patients receiving ArmonAir Digihaler and AirDuo Digihaler.

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Managing your program together (3 of 4)

After the Super Administrator or Administrator on the team has **set up their profile and signed in, they can begin to invite other teammates:**

Now on the “Team List” page, select “Invite Teammate” on the top right side of the screen

Team List

Teammates (6) Pending Invitations (10)

6 Teammates Search list by name

| Last, First | Email Address | Role | Date Enrolled |
|------------------|-------------------------|----------------|---------------|
| Demo, Teammate 1 | demo.teammate1@test.com | Super-Admin | 05/09/2017 |
| Demo, Teammate 2 | demo.teammate2@test.com | Admin | 06/23/2017 |
| Demo, Teammate 3 | demo.teammate3@test.com | Super-Admin | 05/20/2017 |
| Demo, Teammate 4 | demo.teammate4@test.com | Dashboard User | 06/01/2017 |
| Demo, Teammate 5 | demo.teammate5@test.com | Dashboard User | 06/25/2017 |
| Demo, Teammate 6 | demo.teammate6@test.com | Dashboard User | 06/30/2017 |

Remove Remove Remove

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** Other short-acting sympathomimetic bronchodilators should not be used concomitantly with ProAir Digihaler

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Glaucoma and Cataracts:** Long-term use of ICS, including fluticasone propionate, a component of ArmonAir Digihaler and AirDuo Digihaler, may increase the risk for cataracts or glaucoma. Regular eye exams should be considered

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Managing your program together (4 of 4)

After the Super Admin or Admin on the team has **set up their profile and signed in, they can begin to invite other teammates.**

Invite Teammate



This will direct you to the invite form, where you'll be prompted to enter the teammate's email and their assigned role in a dropdown menu

After clicking the "Invite Teammate" button, an invitation link along with the invited Teammate's account email will be generated by the Digihaler® Dashboard. The Dashboard does not send this information. **You will need to copy, paste, and send this information to your teammate via the communication of your choice.** The newly invited teammate will use this information to create their own credentials in the system. If the teammate you've invited has previously completed registration, they will be prompted to join your program immediately following their next sign in.

Please Note:

- The invitation code will expire after 30 days
- To join your program, the teammate must sign up using the same email you entered in the "Invite Teammate" form

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** (continued)
 - » **Beta-Blockers:** Beta-adrenergic-receptor blocking agents not only block the pulmonary effect of beta-agonists, such as ProAir Digihaler, but may produce severe bronchospasm in asthmatic patients. Therefore, patients with asthma should not normally be treated with beta-blockers

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Eosinophilic Conditions and Churg-Strauss Syndrome:** Systemic eosinophilic conditions, such as Churg-Strauss syndrome, may occur when using ArmonAir Digihaler or AirDuo Digihaler. Be alert to eosinophilia, vasculitic rash, worsening pulmonary symptoms, cardiac complications, and/or neuropathy

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Changing the Role of a Teammate

If a user's role needs to be changed, Super Admins and Admins can directly change an existing user's role provided they are not elevating a user beyond their own role. Please refer to [Section 1](#) of this Reference Guide for a breakdown of role responsibilities.

A **Super Admin** can...

- Promote **Dashboard User** to **Admin** or **Super Admin**
- Promote **Admin** to **Super Admin**
- Demote **Super Admin** to **Admin** or **Dashboard User**
- Demote **Admin** to **Dashboard User**

An **Admin** can...

- Promote **Dashboard User** to **Admin**
- Demote **Admin** to **Dashboard User**

To change a Teammate's role, use the drop-down menu to select a different role.

The screenshot shows the 'Team List' interface. At the top, there are logos for ProAir Digihaler, AirDuo Digihaler, and ArmonAir Digihaler. Below the logos, there are tabs for 'Teammates (6)' and 'Pending Invitations (10)'. A search bar is present with the text 'Search list by name'. The main table lists 6 teammates with columns for 'Last, First', 'Email Address', 'Role', and 'Date Enrolled'. The role for 'Demo, Teammate 4' is currently 'Dashboard User', and a dropdown menu is open showing 'Dashboard User' as the selected option. There are 'Remove' buttons for the last three teammates. The footer includes 'Version 4.3.0 © 2022 Teva Respiratory, LLC' and the Teva logo.

| Last, First | Email Address | Role | Date Enrolled |
|------------------|-------------------------|----------------|---------------|
| Demo, Teammate 1 | demo.teammate1@test.com | Super-Admin | 05/09/2017 |
| Demo, Teammate 2 | demo.teammate2@test.com | Admin | 06/23/2017 |
| Demo, Teammate 3 | demo.teammate3@test.com | Super-Admin | 05/20/2017 |
| Demo, Teammate 4 | demo.teammate4@test.com | Dashboard User | 06/01/2017 |
| Demo, Teammate 5 | demo.teammate5@test.com | Dashboard User | 06/25/2017 |
| Demo, Teammate 6 | demo.teammate6@test.com | Dashboard User | 06/30/2017 |

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)


- **Drug Interactions:** (continued)
 - » **Diuretics:** Caution is advised in the coadministration of beta-agonists with non-potassium sparing diuretics (such as loop or thiazide diuretics). Consider monitoring potassium levels


IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Coexisting Conditions:** Use AirDuo Digihaler with caution in patients with convulsive disorders, thyrotoxicosis, diabetes mellitus, ketoacidosis, and in patients who are unusually responsive to sympathomimetic amines

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Managing Pending Invitations

You can view and manage pending invitations for teammates to the Dashboard by clicking the Pending Invitations tab on the Team List page. Click  to renew the 30 day validity period of the invitation, cancel the invitation, or change the role of an invited teammate. This feature is only available for Admins or Super Admins.

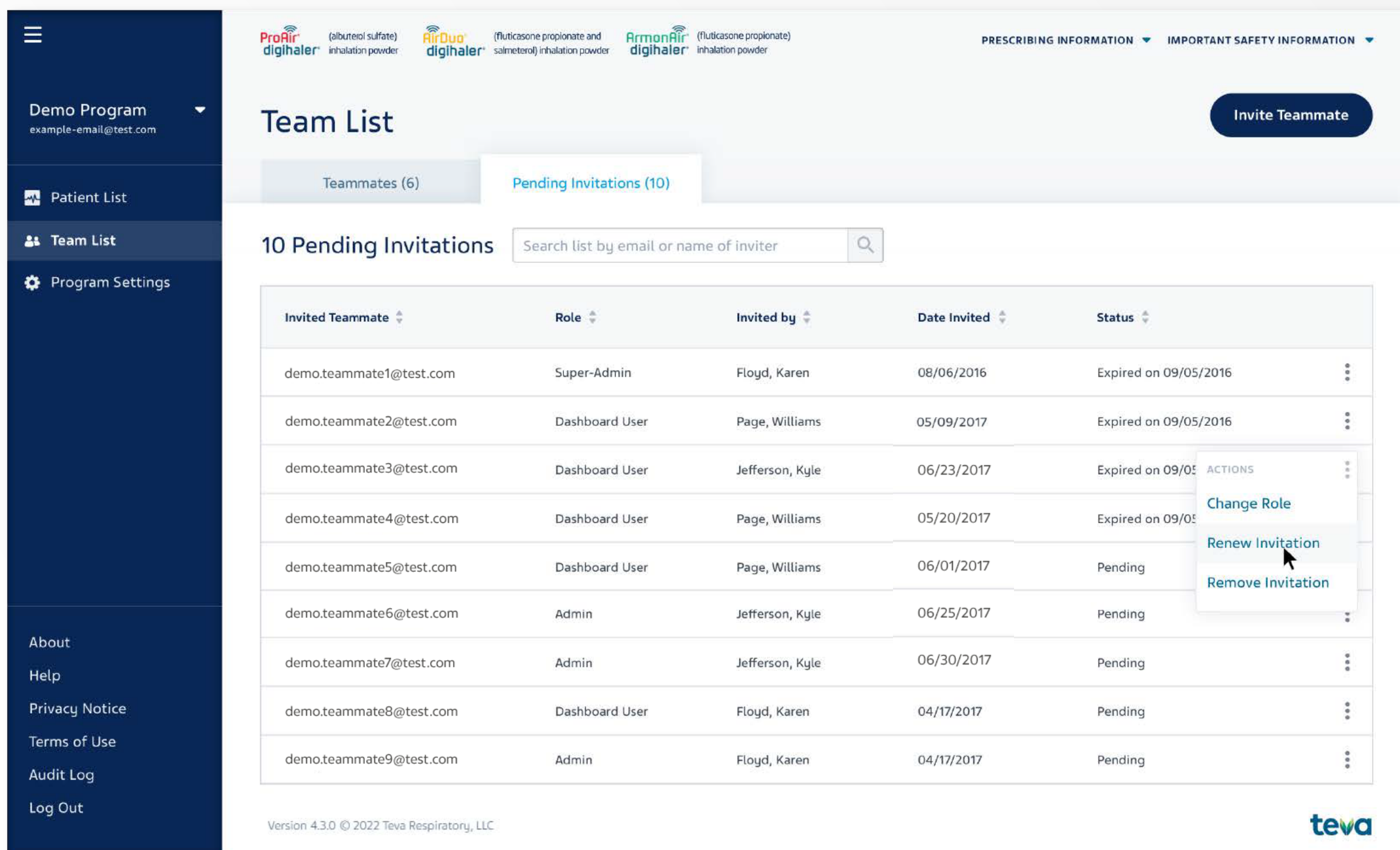

Team List

Teammates (6) Pending Invitations (10)

10 Pending Invitations

| Invited Teammate | Role | Invited by | Date Invited | Status |
|-------------------------|----------------|-----------------|--------------|-----------------------|
| demo.teammate1@test.com | Super-Admin | Floyd, Karen | 08/06/2016 | Expired on 09/05/2016 |
| demo.teammate2@test.com | Dashboard User | Page, Williams | 05/09/2017 | Expired on 09/05/2016 |
| demo.teammate3@test.com | Dashboard User | Jefferson, Kyle | 06/23/2017 | Expired on 09/05/2016 |
| demo.teammate4@test.com | Dashboard User | Page, Williams | 05/20/2017 | Expired on 09/05/2016 |
| demo.teammate5@test.com | Dashboard User | Page, Williams | 06/01/2017 | Pending |
| demo.teammate6@test.com | Admin | Jefferson, Kyle | 06/25/2017 | Pending |
| demo.teammate7@test.com | Admin | Jefferson, Kyle | 06/30/2017 | Pending |
| demo.teammate8@test.com | Dashboard User | Floyd, Karen | 04/17/2017 | Pending |
| demo.teammate9@test.com | Admin | Floyd, Karen | 04/17/2017 | Pending |

Version 4.3.0 © 2022 Teva Respiratory, LLC

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** (continued)
 - » **Digoxin:** Carefully evaluate the serum digoxin levels in patients who are currently receiving digoxin and ProAir Digihaler

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Hypokalemia and Hyperglycemia:** Beta-adrenergic agonist medicines may produce significant hypokalemia in some patients, possibly through intracellular shunting, which has the potential to produce adverse cardiovascular effects. Decrease in serum potassium are usually transient, not requiring supplementation. Be alert to hypokalemia and hyperglycemia in patients using AirDuo Digihaler

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Exporting an Audit Log

If you are looking for a detailed report of teammate activity within your healthcare program, an Admin or Super Admin can choose to export an Audit Log CSV file. With this report, you will be able to view specific actions taken by your teammates including: login, logout, joined program, invited patient, downloaded patient data, etc.

For each action, you will be able to view what the action was, which user took the action, when the action occurred, etc.

The Audit Log is sorted by date and time. When exporting the Audit Log, you will have the option to customize the date range of exported data if you should choose.

The screenshot shows the 'Patient List' page in a healthcare application. The sidebar on the left contains navigation options: Demo Program, Patient List, Team List, Program Settings, About, Help, Privacy Notice, Terms of Use, Audit Log, and Log Out. The main content area displays a table of 10 patients. At the bottom of the sidebar, the 'Log Out' button is highlighted with a callout that says 'Click here to export the Audit Log.'

| Last, First | DOB | ProAir Digihaler (Last 7 Days) | | | AirDuo Digihaler (Last 30 Days) | | | ArmonAir Digihaler (Last 30 Days) | | |
|------------------|------------|--------------------------------|--------|-------------------|---------------------------------|--------|----------------|-----------------------------------|--------|-------------------|
| | | Events | % Good | All Connected | Events | % Good | All Connected | Events | % Good | All Connected |
| Demo, Patient 1 | 08/06/1958 | 5 | 60% | 20 minutes ago | 43 | 93% | 20 minutes ago | - | - | - |
| Demo, Patient 2 | 04/26/1964 | 12 | 83% | 23 hours ago | 33 | 97% | 3 hours ago | - | - | - |
| Demo, Patient 3 | 03/22/1966 | 3 | 0% | a few seconds ago | - | - | - | 78 | 64% | a few seconds ago |
| Demo, Patient 4 | 11/14/1978 | 40 | 38% | 30 minutes ago | 22 | 61% | 1 hour ago | - | - | - |
| Demo, Patient 5 | 02/06/1947 | 17 | 100% | 5 hours ago | - | - | - | 44 | 100% | 5 hours ago |
| Demo, Patient 6 | 01/11/1984 | 4 | 0% | 30 minutes ago | - | - | - | 37 | 97% | 15 minutes ago |
| Demo, Patient 7 | 09/05/1957 | 0 | - | 3 hours ago | 10 | 30% | 3 hours ago | 10 | 30% | 3 hours ago |
| Demo, Patient 8 | 10/24/1999 | 0 | - | 3 hours ago | 10 | 30% | 3 days ago | 0 | - | 3 days ago |
| Demo, Patient 9 | 04/16/1950 | 0 | - | 4 days ago | 10 | 30% | 4 days ago | 0 | - | 1 month ago |
| Demo, Patient 10 | 02/04/1948 | 0 | - | 3 days ago | 0 | - | 2 days ago | 0 | - | 2 months ago |

Click here to export the Audit Log.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Drug Interactions:** (continued)
 - » **Monoamine Oxidase Inhibitors or Tricyclic Antidepressants:** ProAir Digihaler should be administered with extreme caution to patients being treated with these agents, or within 2 weeks of discontinuation of these agents, because the action of albuterol on the cardiovascular system may be potentiated. Consider alternative therapy

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Adverse Reactions with ArmonAir Digihaler:** Most common adverse reactions (greater than or equal to 3%) are: upper respiratory tract infection, nasopharyngitis, oral candidiasis, headache, and cough

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

For Dashboard Admins and Super Admins

Section 8: Program Settings

Program Settings

The Program Settings page allows Admin users to create custom filters that all members of their program can leverage to filter the Patient List.

The screenshot shows the 'Program Settings' page. At the top, there are logos for ProAir Digihaler, AirDuo Digihaler, and ArmonAir Digihaler. A 'Create Filter' button is visible. Below the header, the page title is 'Program Settings' and it indicates '3 Filters'. A table lists the filters with their names and criteria. The filters are:

| Filter Name | Criteria | Edit | Delete |
|--------------------------------------|--|------|--------|
| [Ex: Nighttime Rescue Events] | Digihaler event type(s): [ProAir Digihaler events] Time of day: [between the hours of] Hours: [8:00PM] and [6:00 AM] Date Range: from [1 week ago] to [now] Condition: [is greater than] [2] | Edit | Delete |
| [Ex: Increase in Rescue Events] | Digihaler event type(s): [ProAir Digihaler events] Time of day: [at any time of day] Date range: from [1 week ago] to [now] Condition 1: [compared to] | Edit | Delete |
| [Ex: Decrease in Maintenance Events] | Digihaler event type(s): [AirDuo Digihaler events], [ArmonAir Digihaler events] Time of day: [at any time of day] Date range: from [1 week ago] to [now] Condition 1: [compared to] | Edit | Delete |

Super Admins and Admins will be able to access the Program Settings screen. From here they can add, remove or edit filters.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Paradoxical Bronchospasm:** ProAir Digihaler can produce paradoxical bronchospasm that may be life-threatening. Discontinue ProAir Digihaler and institute alternative therapy if paradoxical bronchospasm occurs

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)


- **Adverse Reactions with AirDuo Digihaler:** Most common adverse reactions (greater than or equal to 3%) include nasopharyngitis, oral candidiasis, headache, cough and back pain

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Configuring Filters (1 of 3)

When defining a new filter:

1. Start by clicking “Create Filter”
2. Give the filter a name
3. Set Filter criteria:
 - a. Choose at least one brand to consider
 - b. Choose a time of day when events may have occurred
 - c. Choose a date range
 - d. Choose the threshold condition (“is greater than”, “is less than”, or “compared to”)
 - e. Choose the numerical threshold
4. Click “Save Filter”

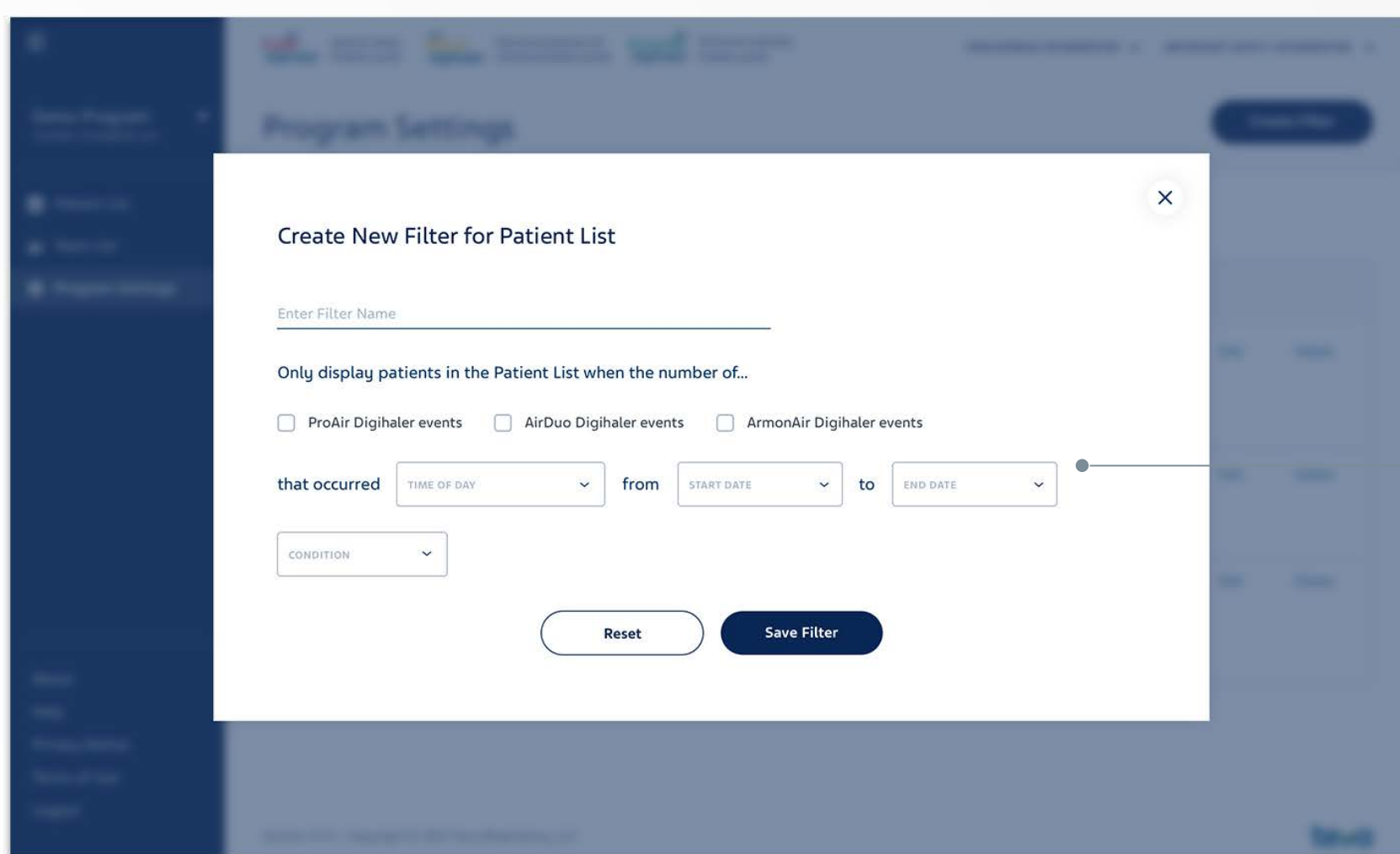

Start by clicking “Create Filter”

Defining certain criteria makes the final decision on whether a patient is included or excluded in the filter.

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Deterioration of Asthma:** Need for more doses of ProAir Digihaler than usual may be a marker of acute or chronic deterioration of asthma and requires reevaluation of treatment, such as possible need for anti-inflammatory treatment, e.g., corticosteroids

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Serious Asthma-Related Events - Hospitalizations, Intubations, Death:** Use of AirDuo Digihaler, LABA as monotherapy (without an ICS) for asthma is associated with an increased risk of asthma-related death. Available data from controlled clinical trials also suggest that use of LABA as monotherapy increases the risk of asthma-related hospitalization in pediatric and adolescent patients. These findings are considered a class effect of LABA monotherapy. When LABA are used in fixed-dose combination with ICS (such as AirDuo Digihaler), data from large clinical trials do not show a significant increase in the risk of serious asthma-related events (hospitalizations, intubations, death) compared with ICS alone

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Configuring Filters (2 of 3)

By selecting the "Condition" drop down and selecting "is greater than" or "is less than", filters can be set up to display patients in the Patient List that are over or under a set threshold, defined by the number of events that occurred within the specified time frame.

In the example below, only patients with AirDuo® Digihaler® Events that occurred within the last month would appear when applying this filter in the Patient List.

Create New Filter for Patient List

Enter filter Name

Only display patients in the Patient List when the number of...

ProAir Digihaler Events AirDuo Digihaler Events ArmonAir Digihaler Events

that occurred from to

Date Range: from 1 year ago to now

Configuring filters allows users to compare:

- how events changed over time from one week to another
- daytime vs nighttime events
- inhaler brands or inhalation categories

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Use of Anti-Inflammatory Agents:** ProAir Digihaler alone may not be adequate to control asthma in many patients. Early consideration should be given to adding anti-inflammatory agents, e.g., corticosteroids

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Deterioration of Disease and Acute Episodes:** AirDuo Digihaler should not be initiated in patients during rapidly deteriorating or potentially life-threatening episodes of asthma. ArmonAir Digihaler and AirDuo Digihaler are not indicated for the relief of acute bronchospasm. An inhaled, short-acting beta₂-agonist, not ArmonAir Digihaler or AirDuo Digihaler, should be used to relieve acute symptoms such as shortness of breath

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Configuring Filters (3 of 3)

By selecting the "Condition" drop down and selecting "compared to", the user can specify a second group of events, and then specify the relationship between the two groups in terms of percent difference.

In the example below, the number of AirDuo® Digihaler® events are being compared from the last month, to the month prior. If the number of AirDuo Digihaler events of the past month are 30% higher than the month prior, the patient is included when applying the filter in the Patient List.

In this example, by selecting the "Condition" drop down and selecting "compared to", you can view how different one group of events are compared to another group of events.

Create New Filter for Patient List

Only display patients in the Patient List when the number of...

ProAir Digihaler Events AirDuo Digihaler Events ArmonAir Digihaler Events

that occurred at any time of day from 1 month ago to Now

compared to the number of...

ProAir Digihaler Events AirDuo Digihaler Events ArmonAir Digihaler Events

that occurred at any time of day from 2 months ago to 1 month ago

is greater than 130 %

Reset Save Filter

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Cardiovascular Effects:** ProAir Digihaler, like other beta-adrenergic agonists, can produce clinically significant cardiovascular effects in some patients, as measured by heart rate, blood pressure, and/or symptoms. If such effects occur, the drug may need to be discontinued. ProAir Digihaler, like all sympathomimetic amines, should be used with caution in patients with cardiovascular disorders, especially coronary insufficiency, cardiac arrhythmias, and hypertension

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Avoid Excessive Use and Avoid Use with Other Long acting Beta2-Agonists:** AirDuo Digihaler should not be used more often than recommended, at higher doses than recommended, or in conjunction with other medicines containing LABA, as an overdose may result. Clinically significant cardiovascular effects and fatalities have been reported in association with excessive use of inhaled sympathomimetic drugs.

Patients using AirDuo Digihaler should not use another medicine containing a LABA (e.g., salmeterol, formoterol fumarate, arformoterol tartrate, indacaterol) for any reason


Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

Using Filters


Once a filter has been configured and saved from the Programs Settings page, it immediately becomes visible to all users of the program. Provided at least one filter has been created, the option to filter will appear on the Patient List.

By default, the Patient List will display an un filtered list with All Patients data. To filter the Patient List, select the Filter button and choose the desired filter from the dropdown menu. Once a filter is selected, the Patient List will adjust based on the filter and the specific inhalations of patient population. An X icon allows users to clear the filter and return to the Patient List containing all patients.

By default, All Patients is selected, which displays the unfiltered list of patients

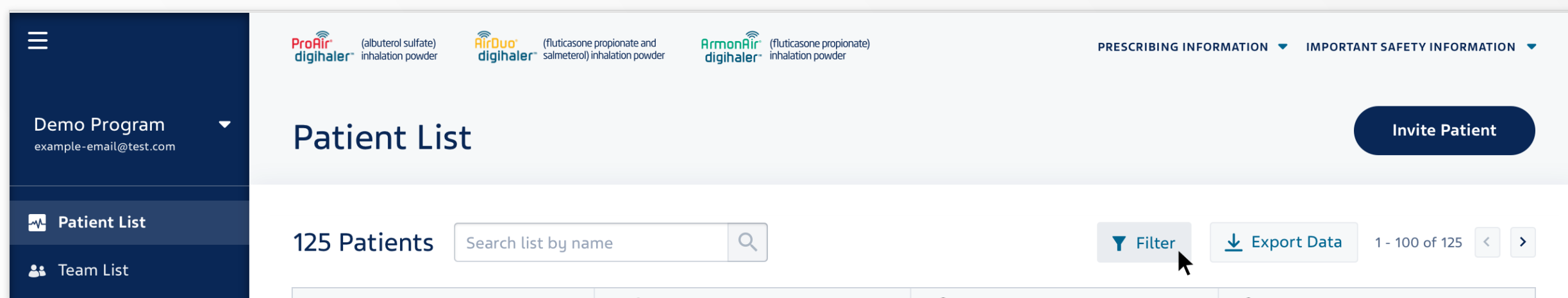

Clicking on the Filter button shows the user what filters are available

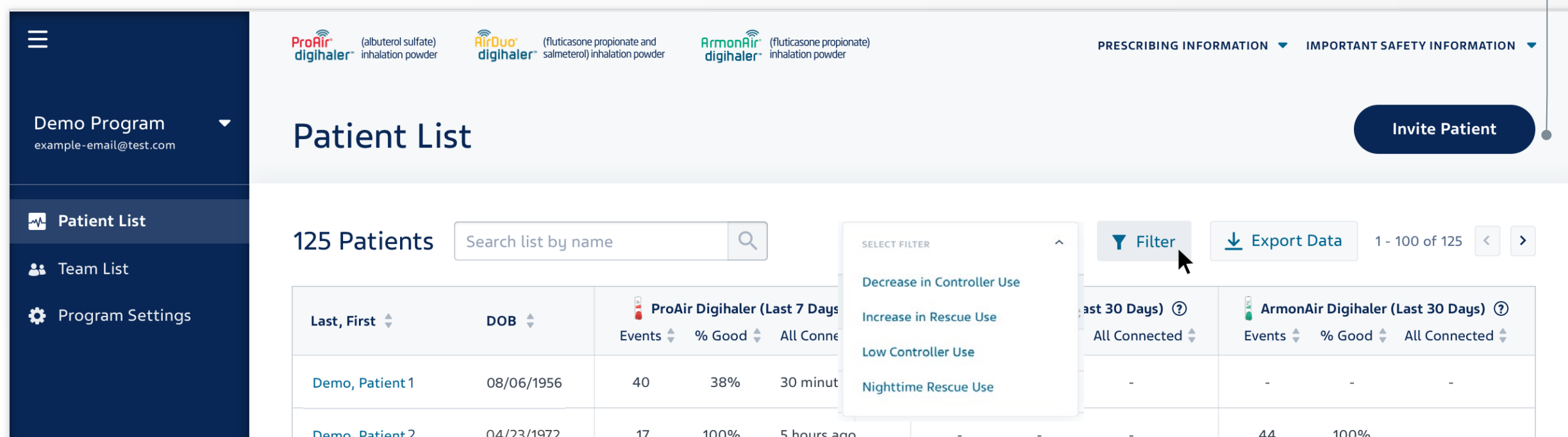

IMPORTANT SAFETY INFORMATION FOR PROAIR® DIGIHALER® (CONTINUED)

- **Do Not Exceed Recommended Dose:** Fatalities have been reported in association with excessive use of inhaled sympathomimetic drugs in patients with asthma

IMPORTANT SAFETY INFORMATION FOR ARMONAIR® DIGIHALER® AND AIRDUO® DIGIHALER® (CONTINUED)

- **Oropharyngeal Candidiasis** has occurred in patients treated with ArmonAir Digihaler or AirDuo Digihaler. Advise patients to rinse the mouth with water without swallowing following inhalation

Please see additional Safety Information throughout this brochure and full Prescribing Information for ProAir Digihaler ArmonAir Digihaler and AirDuo Digihaler.

References:

1. Global Initiative for Asthma. Global Strategy for Asthma Management and Prevention. 2020. <https://www.ginasthma.org>. Accessed on July 7, 2020.
2. Global Initiative for Chronic Obstructive Lung Disease. Global Strategy for the Diagnosis, Management, and Prevention of Chronic Obstructive Pulmonary Disease. 2020. <https://goldcopd.org>. Accessed on July 7, 2020
3. ProAir Digihaler Prescribing Information. Parsippany, NJ: Teva Respiratory, LLC.
4. AirDuo Digihaler Prescribing Information. Parsippany, NJ: Teva Respiratory, LLC.
5. ArmonAir Digihaler Prescribing Information. Parsippany, NJ: Teva Respiratory, LLC.

© 2023 Teva Respiratory, LLC. ADH-41095 08/2023